



HZ-3872C Electric bed user manual

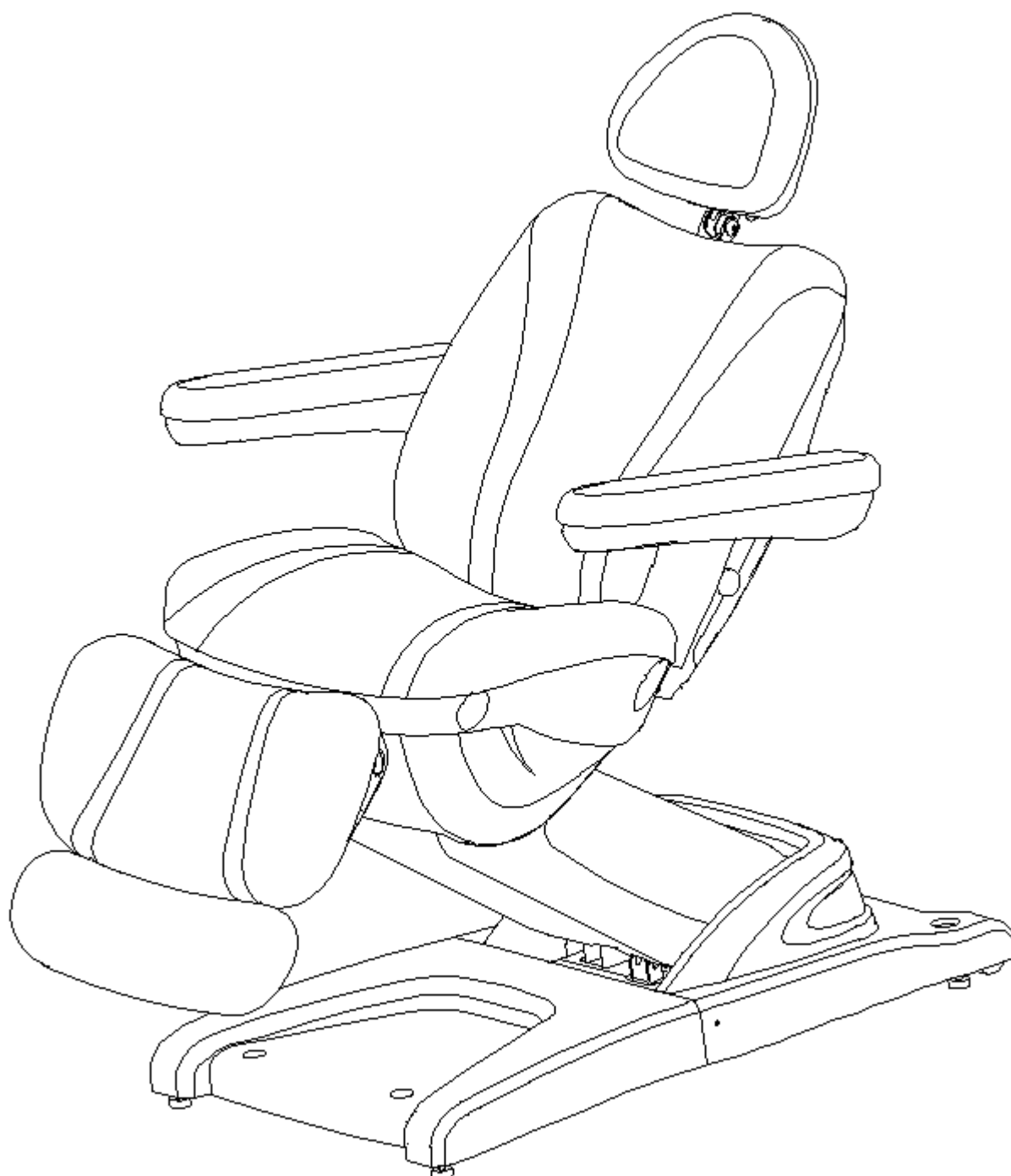

The suggestive instructions:




- Before connection, operation and adjust the bed, please read and the user manual carefully.
- This electric massage bed uses a three-wire power cable. Meanwhile it should be inserted into the three-hole outlet where there is a ground electrode. And it should be assured that it can be connected to the ground electrode smoothly so that it can prevent people from getting electric shock while functioning normally.
- All the plugs, buttons and other components connected to the power cable must not dip in the water or other liquid.
- It is forbidden to put your hands, feet, head or other part of the body into the gap as below position①②③④While adjust the height of the bed by remote control and foot control.
- It is prohibited to use when plugs, buttons and other components connected to the power cable are broken.
- It should be noticed that the frequency is 6min/h when the motor of this massage chair is used.
- The maintenance should not be dealt with without official training.

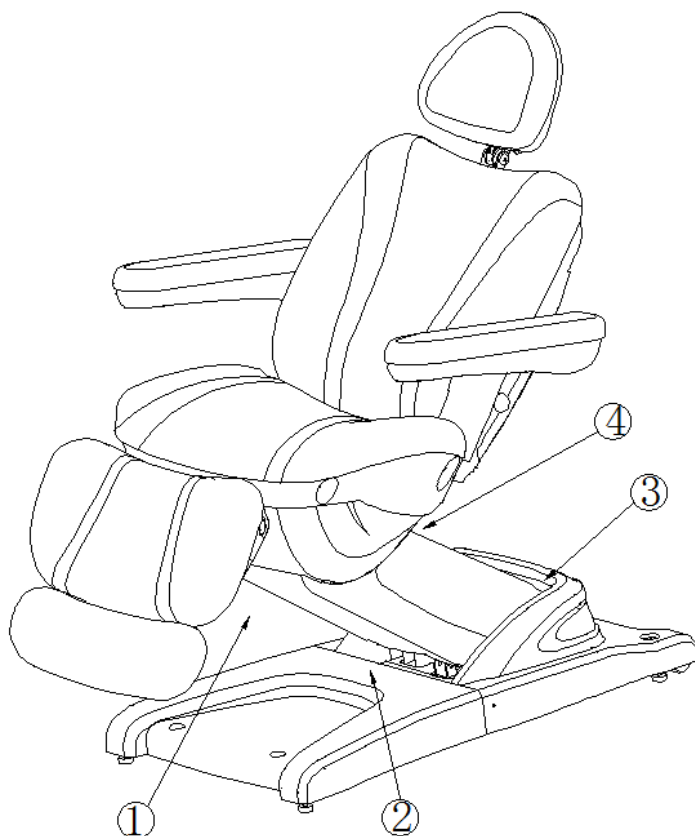

Attentions:

1. Before connection, operation and adjust the bed, please read and the user manual carefully.
2. Women being pregnant or in menstruation should not use this beauty chair;
3. Someone injured or with disabilities should not use this beauty chair;
4. Children under 14 years old without any supervision and someone unconscious should not use this beauty chair;
5. The voltage should accord with the safety voltage before it's officially used. And the wrong voltage may lead to the breakdown of the product;
6. The voltage should accord with the safety voltage before it's officially used. And the wrong voltage may lead to the breakdown of the product;
7. Wet hand is not allowed to touch power to avoid shock;
8. The product should not plug in before it's already installed;
9. The power cable should be put away so that pedestrian will not trip up and fall;
10. The appearance of the product should be cleaned by the use of the soft towel with diluted neutral detergent;
11. It should be unplugged when it's not used;
12. Do not dismount this product without any authorization. Please inform the official service providers to deal with the Maintenance.

● Technical Data:

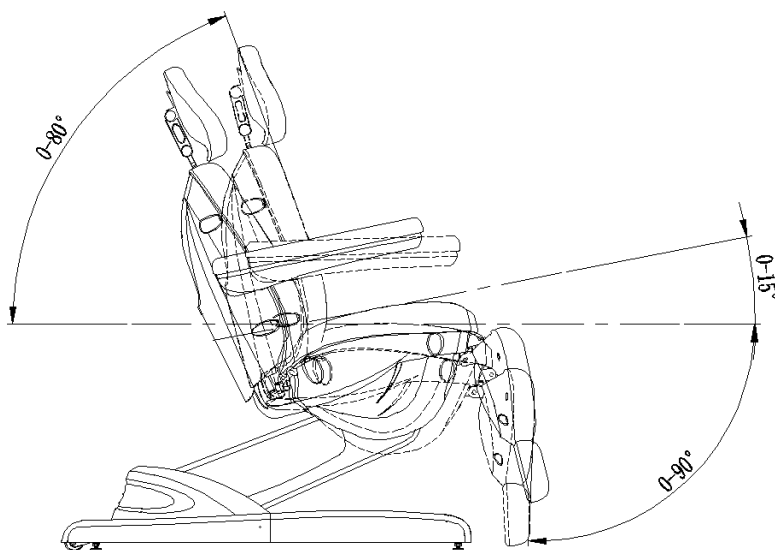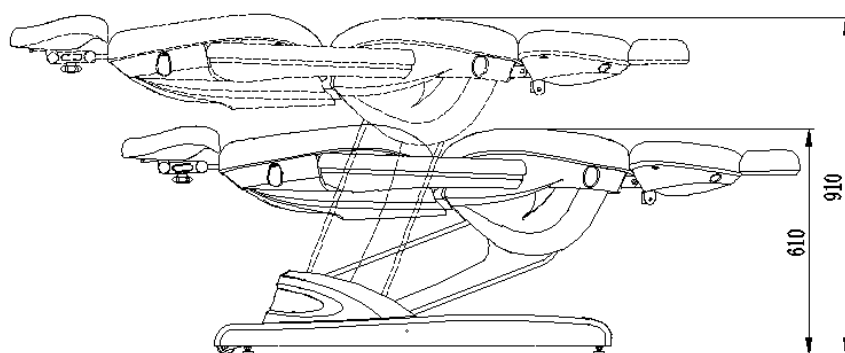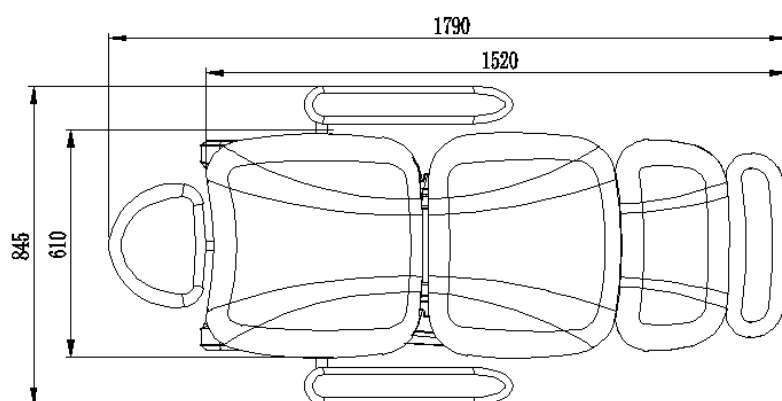

Technical Data:

Size:1790x845x610mm

Min:610mm

Max:910mm

Net weight:73.5kg

Capacity:200kg


Input voltage:
AC100~130V/220~240 ,
50/60Hz

Output voltage:DC 24V

Power rate:110W


● **Operating method:**

1. Assemble headrest. Connect the headrest and backrest, loosen the button to adjust the height of headrest. Such as picture ①.
2. Assemble armrest. Put the armrest to the hole of backrest. connect the groove of armrest and bolt of backrest, and then rotate 90°anticlockwise. Such as picture②.

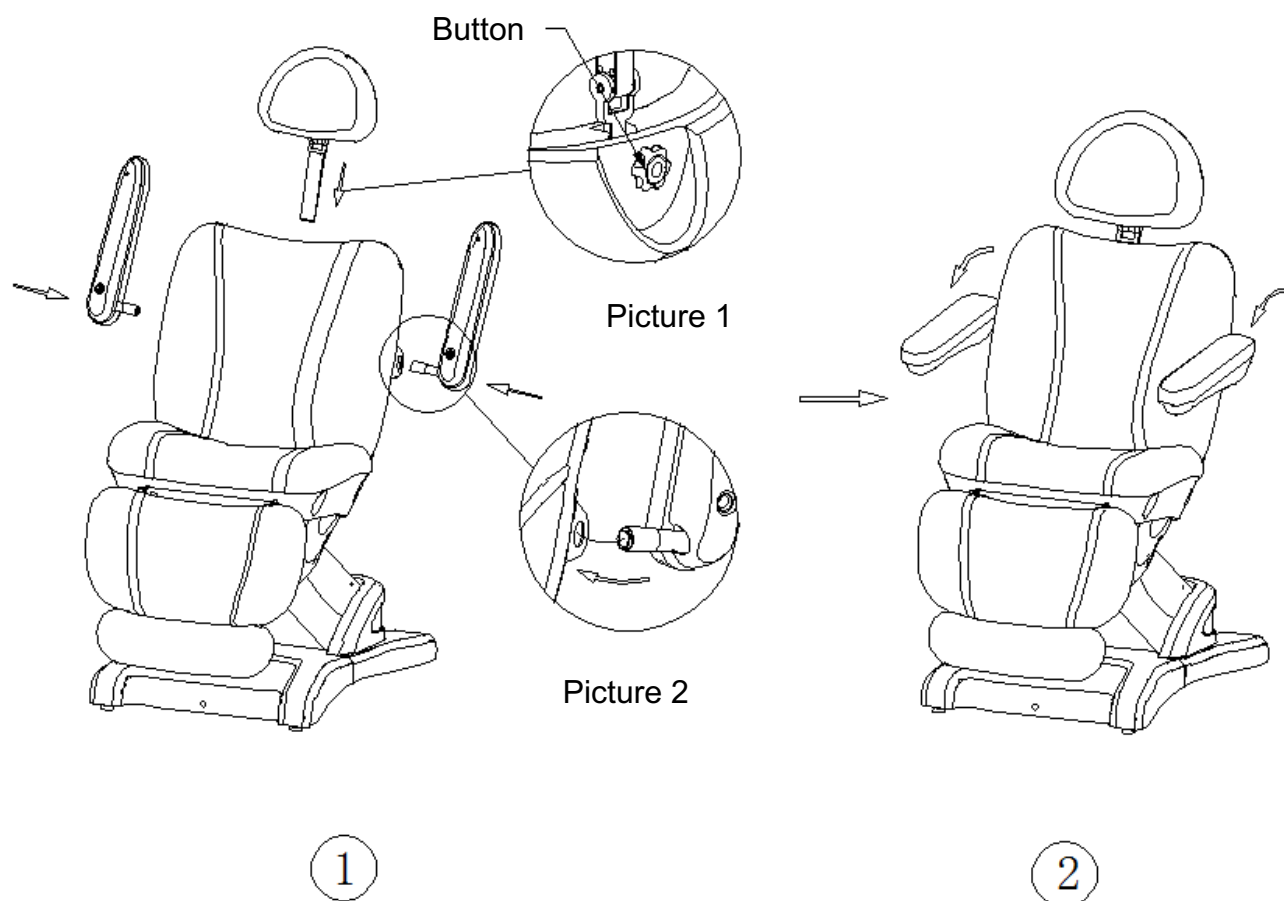

• Controller:

▪ Hand controller:

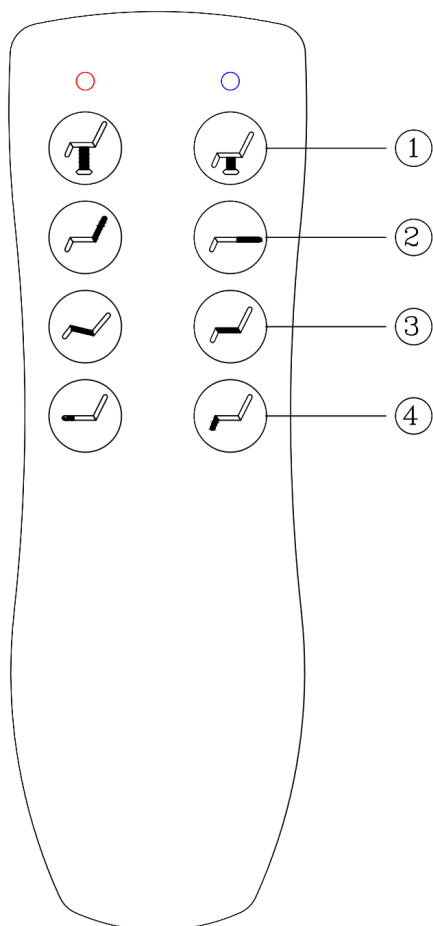

Hand controller:

① SW1: press  or  to adjust the whole bed up and down.

② SW2: press  or  to adjust the backrest up and down.

③ SW3: press  or  to adjust the incline angle for the seat.


④ SW4: press  or  to adjust the incline angle for the footrest.

• Height adjustment:

Turn on the power switch,


press  or  to adjust the whole bed up and down. As shown in the pic:

Power switch

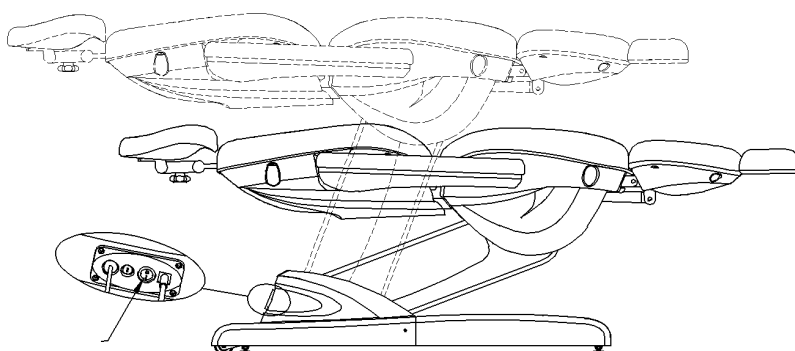


- **Backrest adjustment**

Press  or  to adjust the backrest up and down. As shown in the pic:

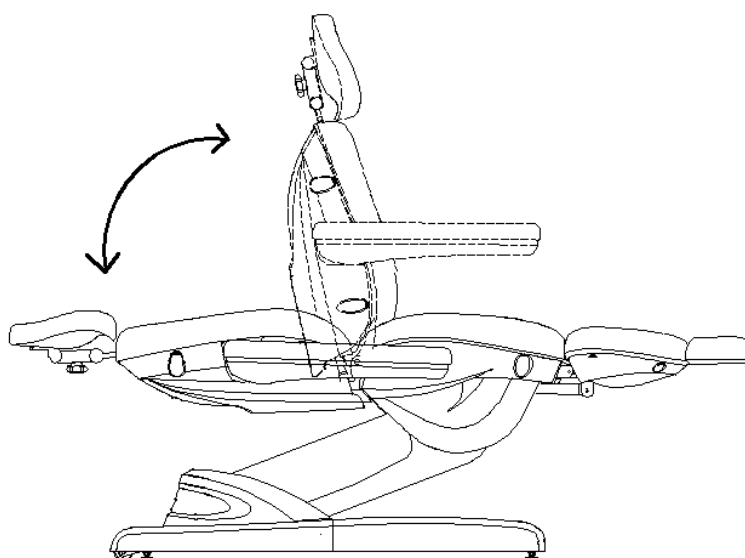


- **Seat adjustment:**

Press  or  to adjust the incline angle for the seat. As shown in the pic:

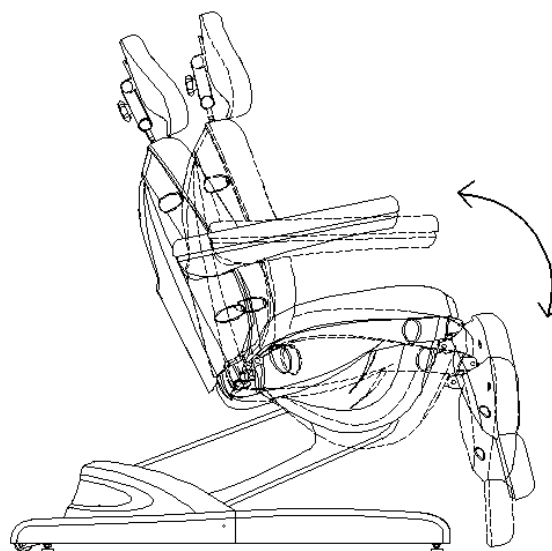

● Footrest adjustment:

Press  or  to adjust the incline angle for the footrest. As shown in the pic:

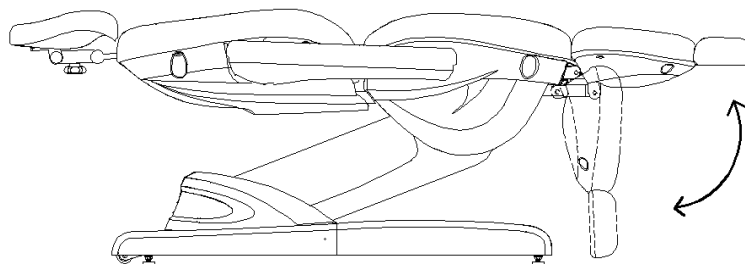


● Pillow adjustment:

Loose knob A to adjust the height of the headrest or assemble and disassemble the headrest,


loose knob B to rotate or flip over the pillow.

As shown in the pic:

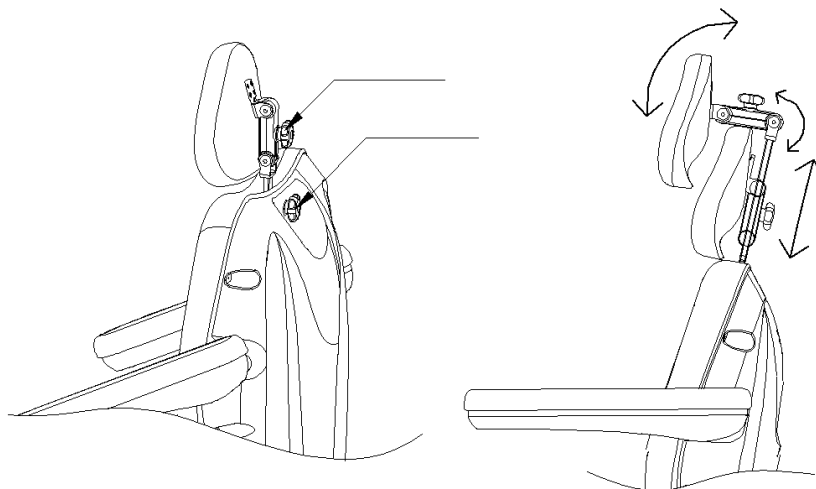


● Footrest extension adjustment:

To adjust the extension of the footrest or assemble / disassemble by rotating the knob under the footrest, as below picture shown.

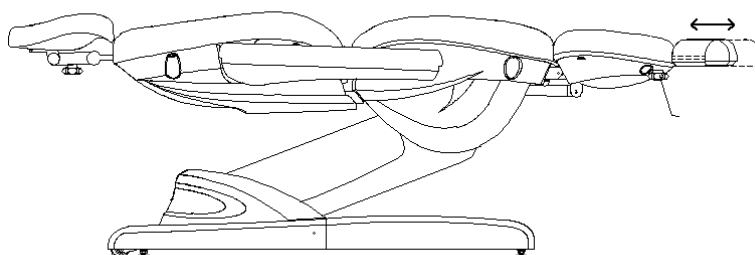

● Rubber foot adjustment

Rotate and adjust height of rubber foot can solve some uneven ground condition, after adjust the height, tighten and fix it upward with the nut, as shown in the picture:

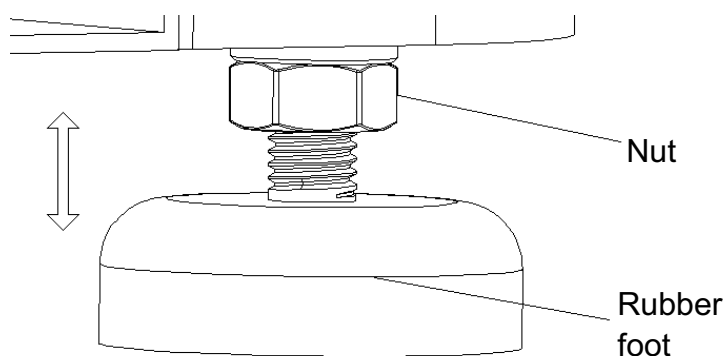

Maintenance:

1. Daily cleaning

You can wipe out the dust by using soft dry towel.

2. Eliminating the stubborn dirt

As we all know, it is hard to eliminate the stubborn dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first. And then you can use dry towel to scrub.

3. It's prohibited to use benzene, diluent, gasoline or other solvent.

These solvents may damage the leather or it may cause the corrosion of the products' appearance.

4. Storage and operating environment

Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause the damages of the product.

5. It is allowed to use this product under the temperature from -5°C to 50°C and the humidity from 45% to 85%

Fault diagnosis:

If there is something wrong when you use the product, please try to check and eliminate the wrong operation as followings.:

Phenomenon	Diagnosis	Operation of elimination
Press the pedal but without any responses.	1.Controller power switch start or not.	1.Try to re-start the controller power switch.
	2.To check if the pedal has reached to maximum.	2.Try other buttons workable or not.
	3. To check if the power plug is connected.	3.Disconnect the power plug and re-plug it again.
	4.To check if each connecting plug	4.Re-plug and connect it

	of the transformer is connected.	again.
Letting out noises and shaking while going up and down	Screw loosen	Tighten all the screws
The light is still off although the switch is on.	The fuse has burned out	Replace the fuse.

◆ **Remarks:** It will be not informed if the improvements of the product do not influence appearance and function