


Instructions Manual

Beautyinstrument

H5050.1/2 H5050.2/2

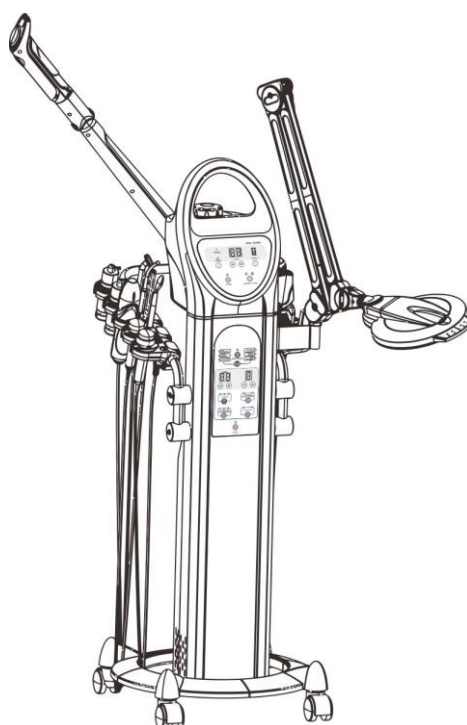

VERSION 1.26

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation
- we shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS


- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

Attention

- Before turn on the power of the host machine, please confirm the water cup was screwed down, the water level was between the highest and lowest water level mark and the water-in cover if tighten.
- If there's needed to add essential oil under working condition, pleas all it after pause the mist spray.
- It must be clean the cotton pallet in the essential oil base after stop using aromatherapy or herbal spray everyday.
- It has to wait the mist came out from the spray hole steady and well-proportionally before facial spray.
- The spray hole is 40cm away from customer's face and spray steam should be from customer's chin to whole face proportionally to avoid scald the customer's face due to hot spray steam.
- Ozone has sterilization effect, but it also will harm skin and destroy nutrients of herbal, so it's important to select suitable time for different skin type when using.

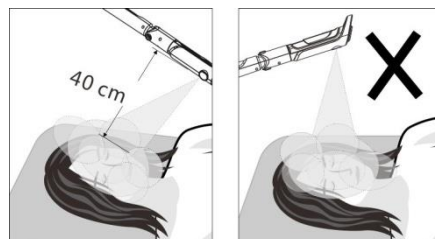

- Do not adjust the spray arm overexertion to avoid damage joints inside the spray arm.
- It's not allowed to take water cup apart when just finish using spray otherwise will cause scald. Please wait water cup cold then take it apart.
- It's necessary to avoid over adding water, otherwise it's not only make bad effect to the host machine, but also may cause the problem of leakage of electricity.
- If there're problems of water leakage and gas leakage happen underneath the host machine or at the herbal medicine cover, remember cut off the power of host machine first and then check whether the water cup or herbal medicine cover well assembled. If the problems still don't be solved, please contact local agency for further solution.
- It should have good ground connection for the power plug of host machine.

Notices for Operation of Spray, Vacuum Functions:

1. Carefully check the label on air flexible tube to prevent incorrect connection.
2. The host should be paused if the spray bottle or sucker needs to be replaced during using.
3. During the working of vacuum or spray function, DO NOT bend the flexible tube or block air inlet, which may directly cause the damage of host.
4. Any crack on nozzle of flexible tube will directly impact on the working effect, to cut the crack part with scissor and re-connects with corresponding joint.
5. It should be noted during operation of spray function:
 - a) Controls the spray mist volume, to prevent the waste of skin refreshener.
 - b) To prevent water drop to neck, where should be protected with water basin or towel.
 - c) **NOT** spray toward naris or mouth, to guarantee the breath.
 - d) **NOT** fill in spray bottle with liquid with low viscosity, to prevent any block.
 - e) The spray bottle should be cleaned in time with distillate or purified water after each time use, and wipes dry and sterilizes for next time use.
6. It should be noted during operation of vacuum function:
 - a) The skin parts should be cleaned and softened before operation of vacuum function.
 - b) The sucking intensity of vacuum tube should be controlled medially, high intension could cause skin lesion and coagulation of blood below the surface of the skin. The beautician firstly could try by own hands, and moves to working parts after adjustment of vacuum ability.
 - c) When the treatment is made on fix parts of skin, the press time of relief-pressure air inlet should be controlled by fingers, to prevent too long time sucking and become purple by sucking.
 - d) The sucking frequency should be increased for oily, thick skin, and sucking is not allowed to be used for skin with telangiectasis.
 - e) **DO NOT** use for eye skin or face with inflammation.
 - f) **DO NOT** use frequency the vacuum sucking, to prevent enlarge of skin pore.
 - g) The vacuum bar should be uninstalled in time after use, and clean, wipe dry and sterilizes for next time use.
 - h) The clean sponge filter should be replaced after each time using, **DO NOT** use vacuum bar without sponge filter, which will directly damage the host.

Notice for Ultrasonic Wave Operation:



1. The probe should be kept clean; clean and sterilize the probe with ethanol or other agents after uses.
2. **DO NOT** use the probe toward eyes when uses ultrasonic wave.
3. The skin care essence should be supplemented in time if insufficient to moisturize the skin.
4. The treatment time should not more than 15min. and should be shorten appropriately for the small area to be nursed, too long time would cause tired of skin and is not good to absorb. The probe should not stay a long time on one part of skin area to prevent any skin lesion.
5. During the course of treatment, it should prevent a long time treatment under the condition of maximum energy output which will not only impact on the absorption of skin but even cause skin lesion.

Notice for Operation of Galvanic Zwitterions Conduction:

1. The zwitterions conduction instrument is normally used after cleaning of skin and before the massage. Before treatment, the face should be cleaned, dry, as to prevent bringing any residual cleaning agent into skin basement.
2. The probe is rolled tightly with cotton pellet, and the current is adjusted from weak to strong for gradual

adaption, then to keep always a light stinging sensation.

3. During the treatment course, the hands of the beautician **DO NOT** touch any metal part of probe.
4. The conduction is applicable to dry skin (insufficient water), decrepitude skin (insufficient nutrition) and oily skin. The different liquid beauty products could be selected according to different skin types, but the beauty product for oily skin will prevent permeating.
5. After operation, clean the roller, metal bar, and keep in dry place.
6. When the polar is set up for positive electrode, the stinging sensation of roller probe will be felt stronger than set up for negative electrode and weaker on the contrary. Please select as your demand.
7. During the treatment course, it is not allowed to wear any metal ornament or jewelry, and the client with gold teeth or metallic contained in the body is not allowed to use.
8. The client with sensitive skin, injury, inflamed, or telangiectasia and pregnant women is not allowed to use this instrument.
9. The **pH** of nutrient solution should be appropriately matched with positive and negative electrode of rollers, when the roller is in positive electrode, the conductioned solution should be acidity (+); when the roller is negative electrode, the conductioned solution should be basicity (-), if the pH of used nutrient solution has not indicated, pH test paper could be used for determination.
10. During the operation, **DO NOT** directly contact with metal part of roller or round probe, if contacts by accident, the host will activate auto-protection, should separate the probe at once, and the host will cancel the protection and continues for treatment.

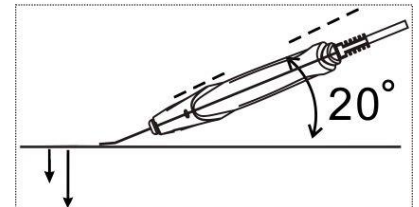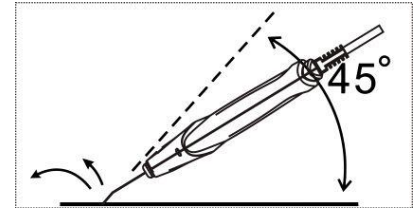

●**Attention:**

- 1、Please do not place the working head of scraper knife on skin for a long time, in case of hurt skin.
- 2、Please pay attention to the using strength when doing skin care, place the scraper knife 45 degree with skin is the best (As pictures shown at right side).
- 3、Please supply essence to surface skin to keep moisture.
- 4、Please keep scraper knife 20 degrees with skin when doing import massage (As picture shown at right side) .
- 5、Do not over pull the cable of accessories to the host machine.
- 6、Please take scarper knife gently when using it, and clean it after every time using, but do not put it in the water to clean. After cleaning, please wipe dry and sterilized, keep clean and dry.
- 7、Please clean and sterilize the sensor after every time using, keep it clean and dry.
- 8、If the cable is destroyed, we suggest using the accessories from repairing agency or original factory.
- 9、NOT allow to be no operation under input state of scraper knife, as this can cause the scraper knife to overheat and rapidly reduce its service life and risk scalding the skin!

●**Attention:**

1. During setting or move magnifying lamp, don't let fingers in the middle of the upper and lower arms for avoiding clamp.(As picture shown)
2. Clean skin before operation.
3. Turn off power after operation.
4. Please cut the power if there's a long time not to use, make sure the machine clean and keep it in a suitable place.

Notices for the using functions of the host machine:

1. During working, the cables between accessory and host machine shall not be drawn excessively.
2. Broken mains cord shall be replaced with specific cord or specific assembly from its manufacturer or maintenance departments. It is strongly recommended that the accessory from original factory shall be used.
3. The host machine shall be turned off and the attaching plug shall be pulled out if any abnormal condition occurs. You shall ask the agency or specific serviceman to check and maintain. The main case is not allowed to be opened.
4. It is forbidden to place or use it at bathroom of wet places.
5. It is forbidden to put it together with inflammable and explosive objects.

WARNING

When using the hot spraying function :

1. It's forbid to put oily material into water cup or herbal net; it may cause the problem of water spray.
2. It's **not allowed** spray from the above of the face by the steam hole, otherwise may causing scald.
3. It's **not allowed** to put this machine in the over humidity environment to work, for example: bath room etc.
4. It's **not allowed** working under projective sunshine, otherwise will cause many kind of unsteady conditions.
5. If there's any exceptional problem, please contact the agency to let professional maintainer to repair instead of opening the host machine cover to repair by you.
6. If power cord is destroyed, it's better to choose the accessories from the original factory. It's not allowed to repair by un-professional personnel; you should go to appointed repair agency to change.

When using high-frequency electrotherapy function:

1. During using of high-frequency electrotherapy, it is not allowed to contact any metallic, and should be taken off if wearing.
2. The client with sensitive skin, patch skin, patient with acne rosacea, wounded skin surface, pregnant women and body equipped with other instrument (e.g. cardiac pacing instrument) **CAN NOT** use this function.
3. When using high-frequency electrotherapy, **CAN NOT** use creams and face mask with content of ethanol. If it has to use toner with ethanol content, it can be used after the high-frequency electrotherapy.
4. **DO NOT** use each electrotherapy tubes toward eyeball.
5. **DO NOT** touch high-frequency electrotherapy main input port or handle input port under condition of live host.
6. The plug of accessories flexible cable should be fixed firmly with corresponding outputs of host.
7. This instrument should be operated by professional beauticians, because high-frequency interference will be generated during accessories working, **DO NOT** close to other electronic instruments. **BE CAUTION** during using.

When using Galvanic Zwitterions Conduction Function:

1. The patients with following disease are not suitable to use this function, e.g. heart disease, infectious disease, neuroticism and body with metallic during cure course.
2. **DO NOT** make the working conduction and export probes touch together for a long time.
3. When the skin care essence is insufficient to moisturize, it should be added in time.
4. **DO NOT** use sliding roller which can not rotate, and should be replaced in time.
5. Prevent spray water or other corrosive liquid on the host during using.
6. **DO NOT** use round probe and roller probe with damaged surface, rusty or worn, to prevent injury of skin.
Be careful to protect the probe, to prevent any scarp by other hard matter.

When using Ultrasonic Function:

- 1、Not allow the probe under output status without facial treatment, overheating is shorten its' service life.
- 2、Do not try to apart the stainless probe and handle, it will damage the probe.
- 3、Prohibit to put the probe into water for cleaning, it will damage the probe.
- 4、Must be change the probe if it damage or surface scratched.

















When using Rotary brush function:

- **Do not** use it for a very long time, otherwise it will cause brush handle over hot.
- **Do not** change brush during it is working.
- **Do not** grasp the brush when it is under working or when you start this function.
- **Do not** use crystallized brush which has abrasion.

When using Spray, Vacuum Function:




1. Be care of flexible tube connecting accessories and host, **DO NOT** over-pull.
2. Prevent spray water or other corrosive liquid on the host during using.
3. Forbidden connecting the flexible tube of spray function with output of vacuum function (see the Fig. right) as well as connecting of flexible tube of vacuum function with nozzle of spray bottle (refer to the Fig.), otherwise it will suck back the treatment liquid into spray bottle, and directly damage the host.
4. Forbidden using sucker with edge angle, sharp edge, worn and damaged. In this case, please contact with distributor or appointed maintenance center to replace in time.

●Packing list:

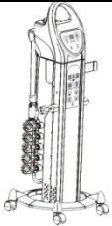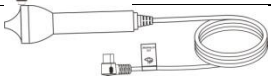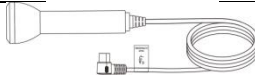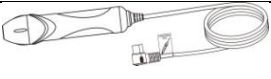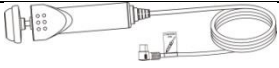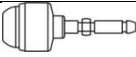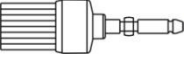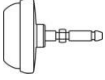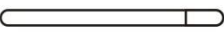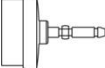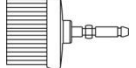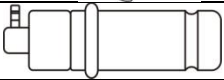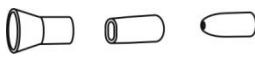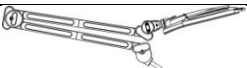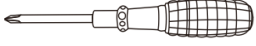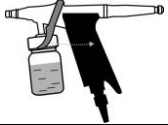Item	Product Description	QTY	Item	Product Description	QTY
Host machine		1 SET	Roller probe		1 pcs
Power cable		1 pcs	Eyes probe		1 pcs
Face probe		1 pcs	High-frequency Electrotherapy Handle		1 pcs
Brush handle		1 pcs	Mushroom-shape Glass Electrotherapy Tube		1 pcs
Crystallized small brush		1 pcs	Spoon-shape Glass Electrotherapy tube		1 pcs
Fuzz small brush		1 pcs	Dot Glass Electrotherapy Tube		1 pcs
Crystallized big brush		1 pcs	Straight-Bar Glass Electrotherapy Tube		1 pcs
EVA big brush		1 pcs	Suction hose tube		1 pcs
Fuzz big brush		1 pcs	Spray hose tube		1 pcs
shag big brush		1 pcs	straw		1 pcs
Spray bottle		2 pcs	Sponge filter		3 pcs
Sucker		1 pcs	Skin Scraper Knife		1 pcs
Magnifying lamp		1 pcs	Instruction Manual		1 copy
Nozzle sleeve		1 pcs	screwdriver		1 pcs
Spray		1 pcs			

Host Machine instruction

- | | |
|-----------------------|-------------------------------------|
| 1. Spray arm | 2. Magnifying lamp |
| 3. Control panel | |
| 4. Accessories holder | 5. Power connection |
| 6. Water-in drawer | |
| 7. Magnifying holder | 8. Spray and vacuum function output |

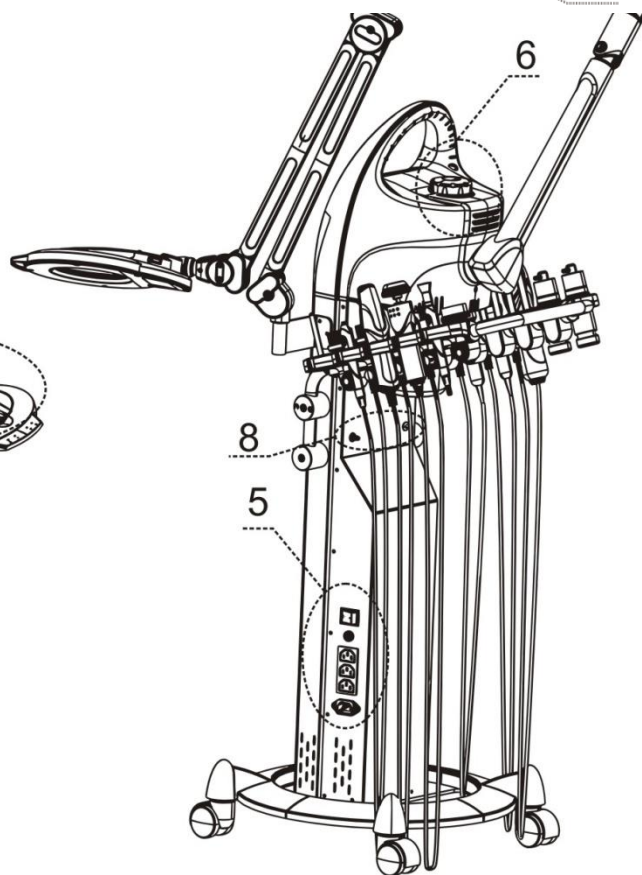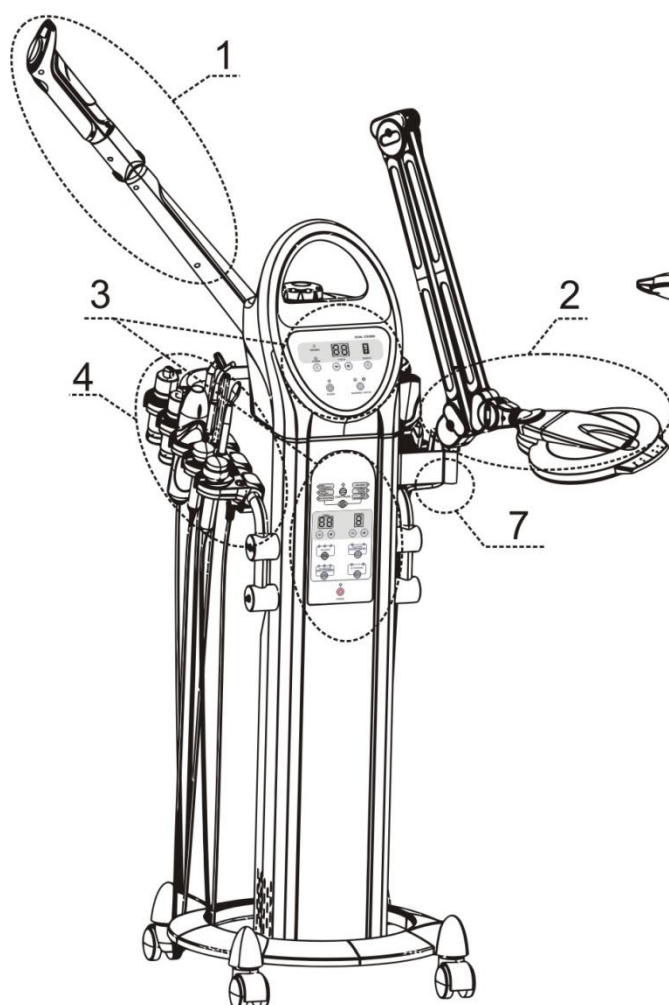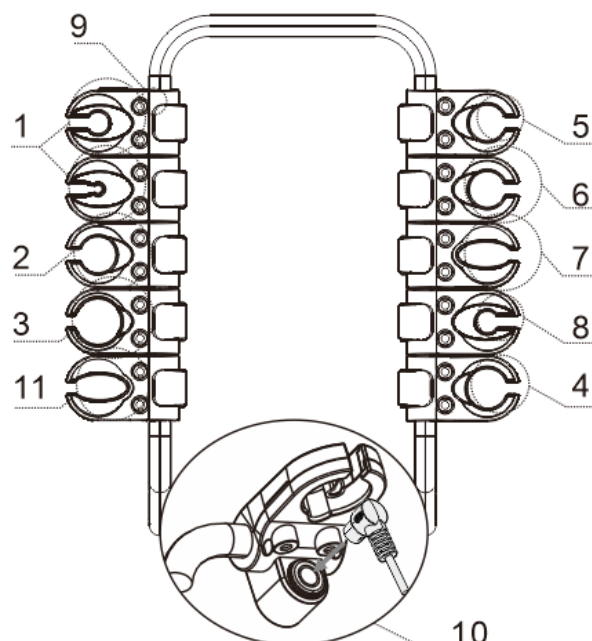


1. Galvanic probe
2. Brush handle

- 3.High frequency handle
- 4.Spray bottle
- 5.Face probe
- 6.Eyes probe
- 7.Skin scrubber knife
- 8.vacuum bar
- 9.Glass Electrotherapy Tube , brush holder
- 10.Accessory output connection
- 11.Spray gun


1. Main power switch
2. Fuse holder
3. Power output socket
4. Power input socket

Accessories


●Ultrasonic:

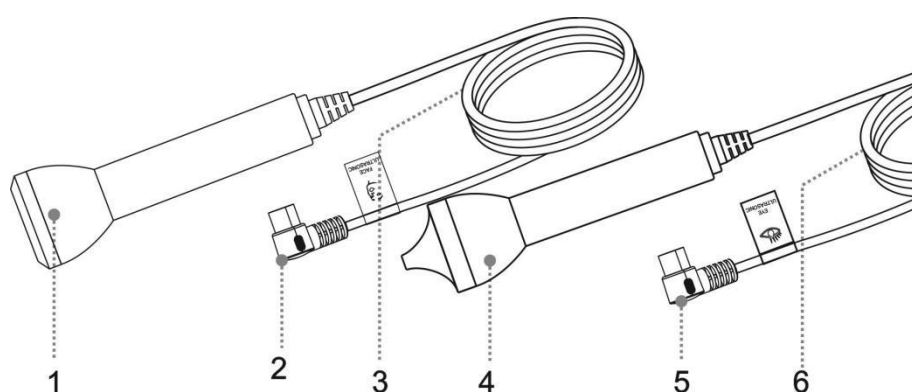

1. Face probe
2. Plug for Face probe flexible cable
3. Face probe flexible cable
4. Eyes probe
5. Plug for Eyes probe flexible cable
6. Eyes probe flexible cable

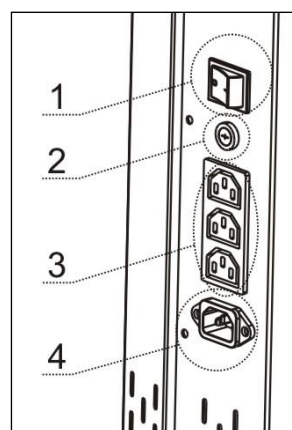


●Skin Scrubber:

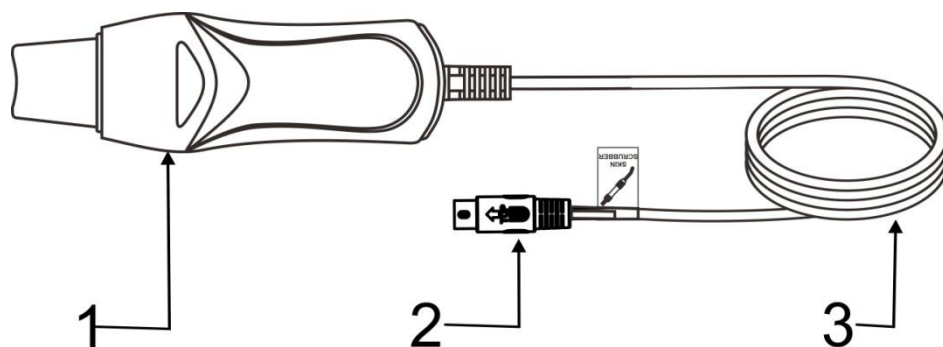


1. Skin Scrubber
2. Plug of Scraper Knife Flexible Cable
3. Flexible Cable of Scraper Knife


●Galvanic:

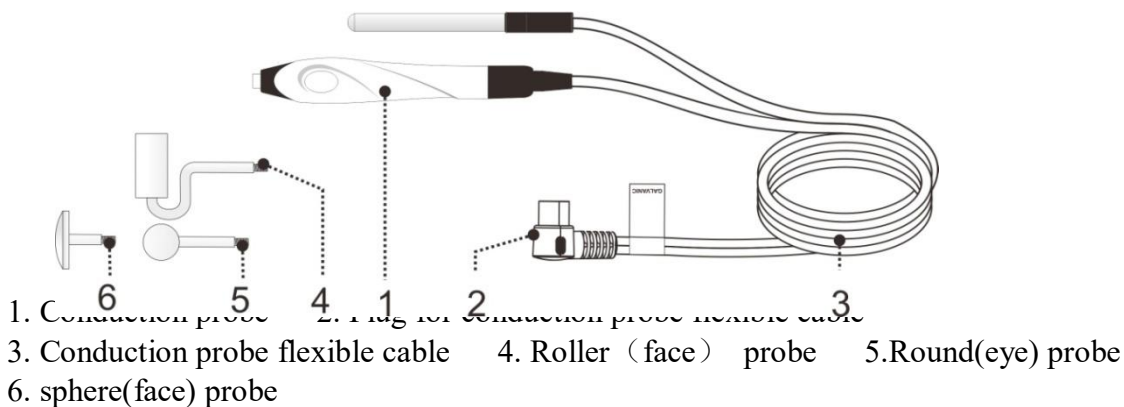


●High-Frequency Electrotherapy:

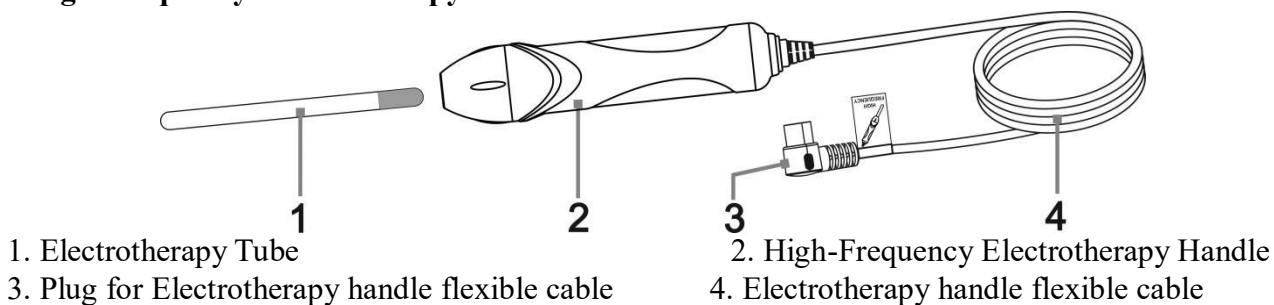

Different glass electrotherapy tubes are suited for treatment of different parts:

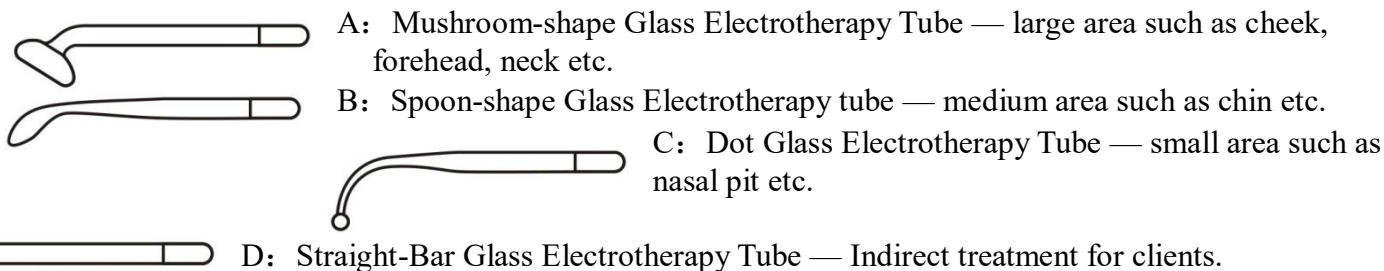

Assembling and Disassembling of Electrotherapy Tube:

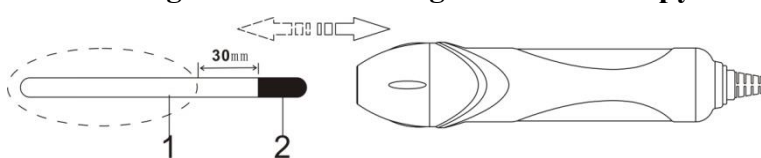


1. Glass part of Electrotherapy tube
2. Metal electrode of Electrotherapy tube

During the assembling and disassembling of electrotherapy tube, hand should hold the parts within dotted line and keep distance with metal part more than 30mm.

●Brush handle


1. Brush head
2. Brush handle
3. Brush flexible cable plug
4. Brush flexible cable

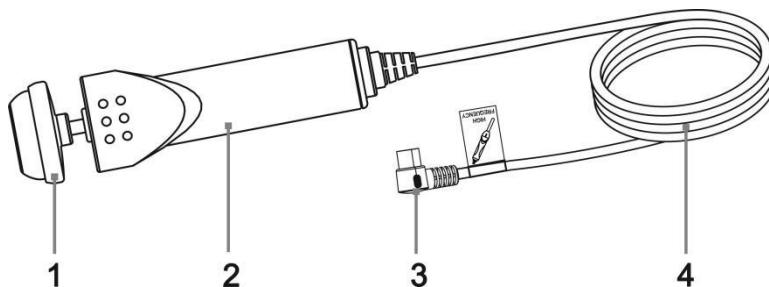


Brushes

1. Crystallized small brush: Clean small area horny and callus of hands and feet.
2. Fuzz small brush: Clean and massage small area skin of face etc..
3. Crystallized big brush: Clean large area horny and callus of hands and feet.
 - a) EVA big brush: Clean and massage large area skin of face etc.
4. Fuzz big brush: Clean and massage large area skin of face etc.
5. Shag big brush: Clean and massage large area skin of face and other position.

●Spray-Vacuum accessories :

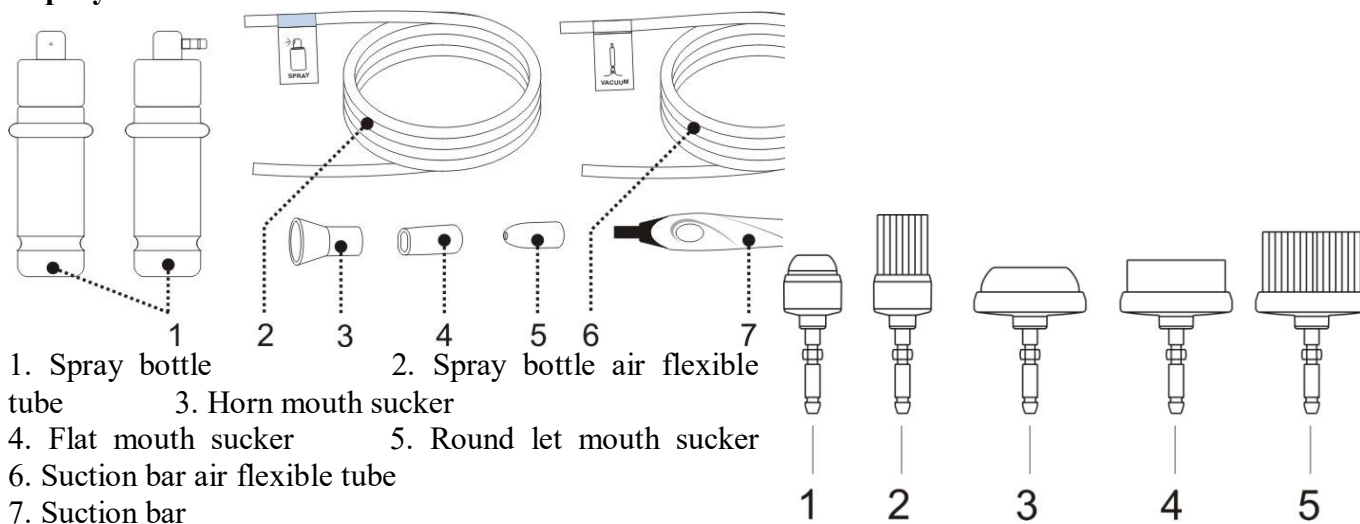


● Spray bottle:

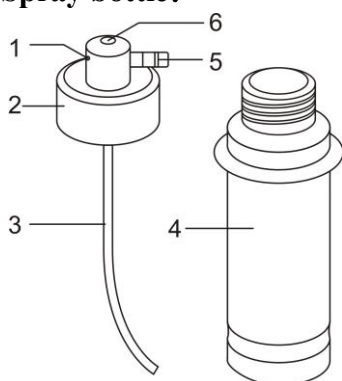


- 1.Nozzle
- 2.Cover of bottle
- 3.Suction tube
- 4.Spray bottle
- 5.Air inlet joint
- 6.Air vent

●Spray

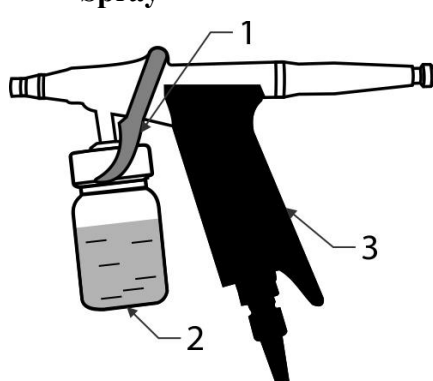

1. Trigger (control the volume of the mist)
2. Bottle
3. Handle


●Suction Bar:

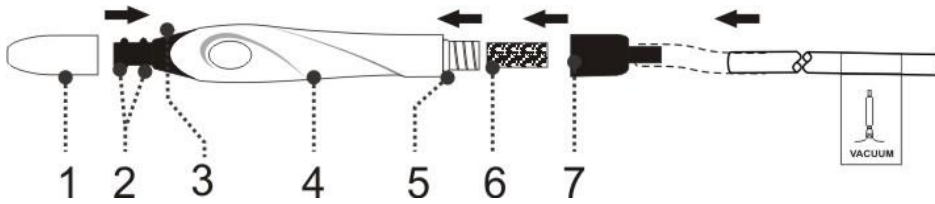


1. Round let mouth sucker 2. Seal ring of sucker 3. relief-pressure air vent
4. Suction bar 5. Seal ring of bar 6. Sponge filter 7. Suction bar end

Preparation:


1. Remove the main machine and the base from the carton, align the 4 screw holes of the base to the main machine (see the dotted circle), tighten with screws.
2. (as direction of picture) to Insert the steamer and compress it, make sure the power outlet is in good contact.

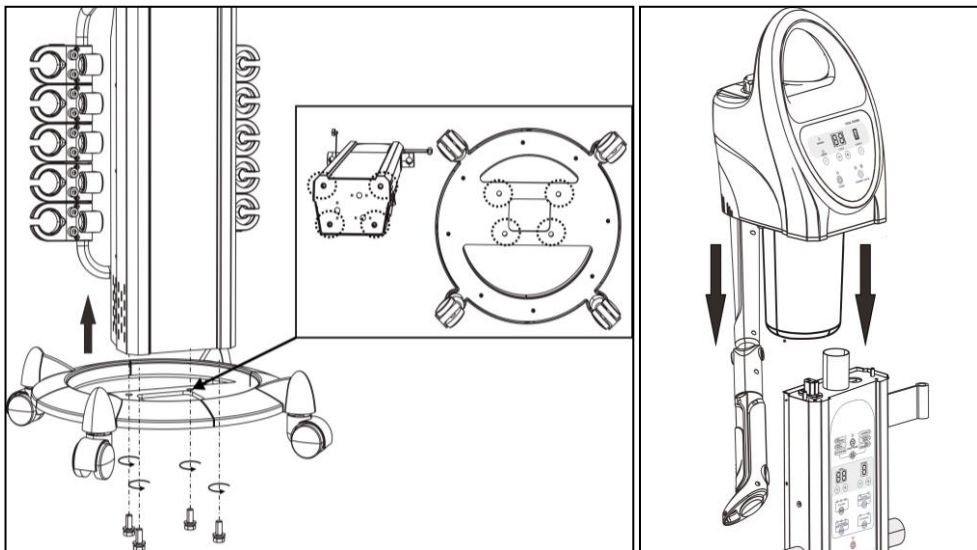


3. Spray arm unlock: when take out the whole machine for the packing ,the spray arm was locked in fixed position, Please pull out the limited lock with one hand (as direction of the black arrow in the picture),Rotate the spray arm to horizontalposition (as following picture), release the limited lock for spray arm to finish the spray arm unlock.
4. Rotate upward the accessory holder (as direction of picture)
5. Install the fixed component, Clamp the pipe with component, 1 and 2,then tighten with screws (3)

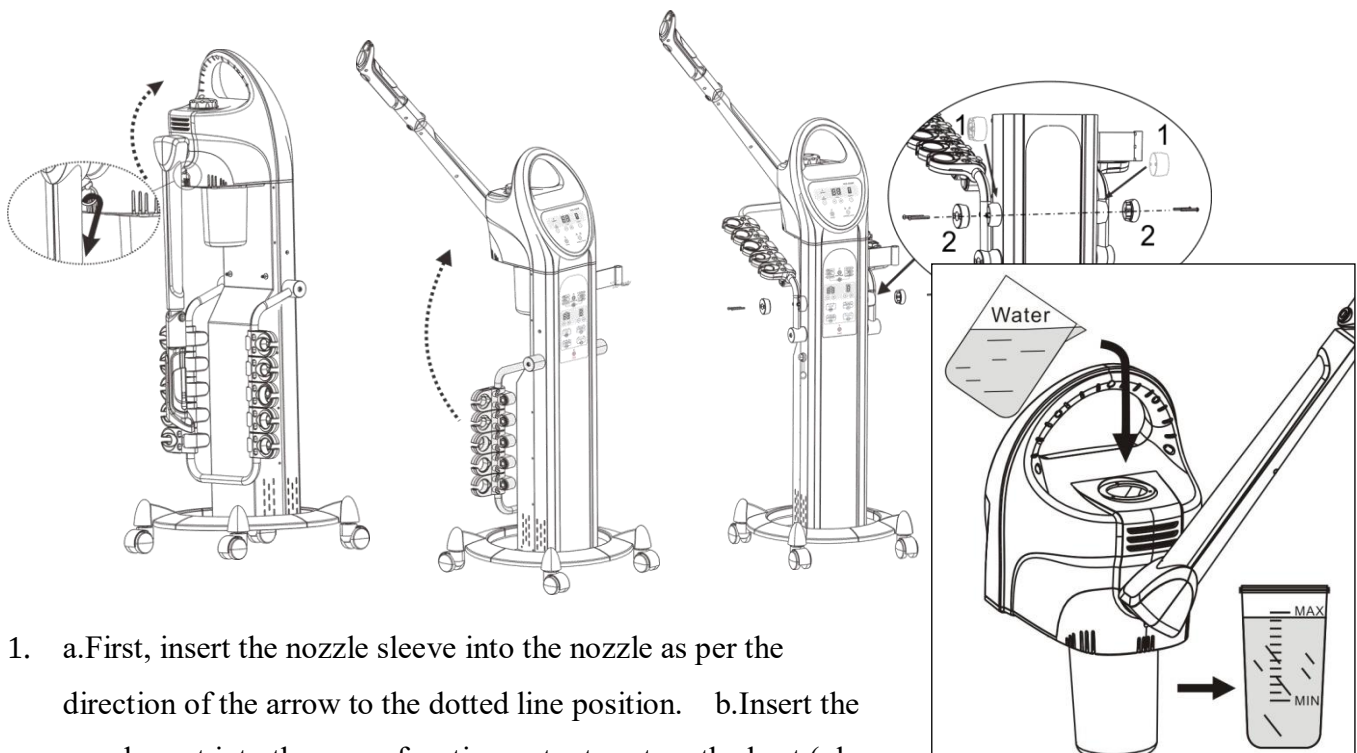

1. a.First, insert the nozzle sleeve into the nozzle as per the direction of the arrow to the dotted line position. b.Insert the nozzle port into the spray function output port on the host (please note: the output port needs to be completely wrapped).c.Finally, put the nozzle sleeve into the spray function output port in the direction of the arrow (dotted line position), and fix the nozzle interface. The spray bottle is connected to the other end, and the installation of this function is completed.

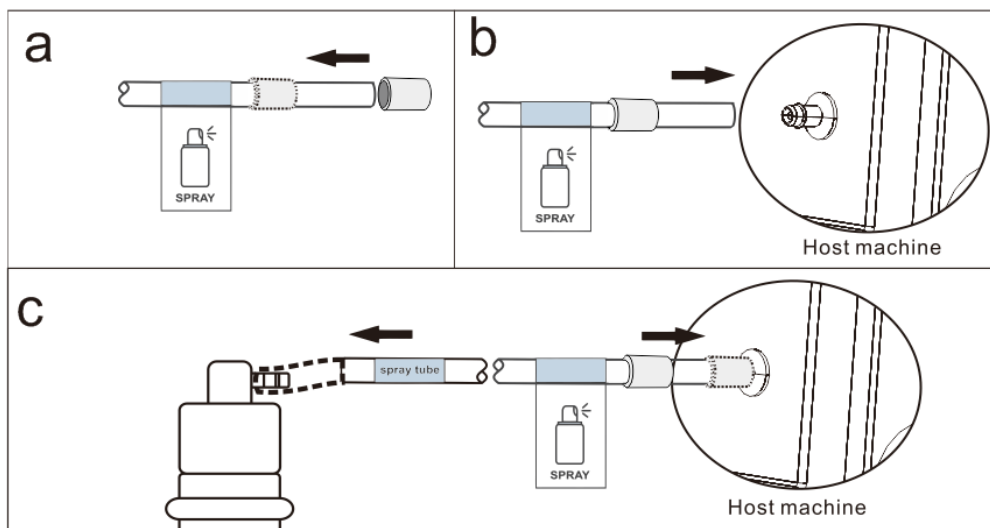

- 2.Insert effectively one end of air Flexible tube with labeling of "**vacuum tube**" to inlet of suction bar, and another end to vacuum function output of host machine

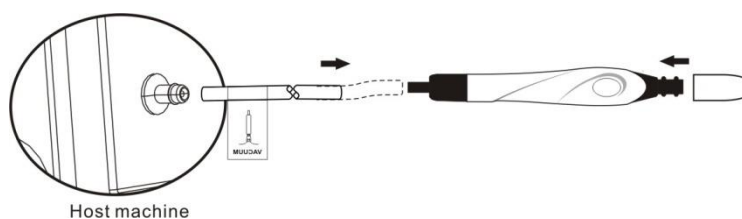

- 3.Put the water into the cup


Open the water-in cover, put the distill water or purifiedwater into the drawer with measure cup.

The water level must be between “MAX” line and “MIN” line.Effective capacity: 520 ml.

Minimum water level capacity: 280 ml. (check the follow picut

Instruction of Functional Operations & Notices:

Hot spray function

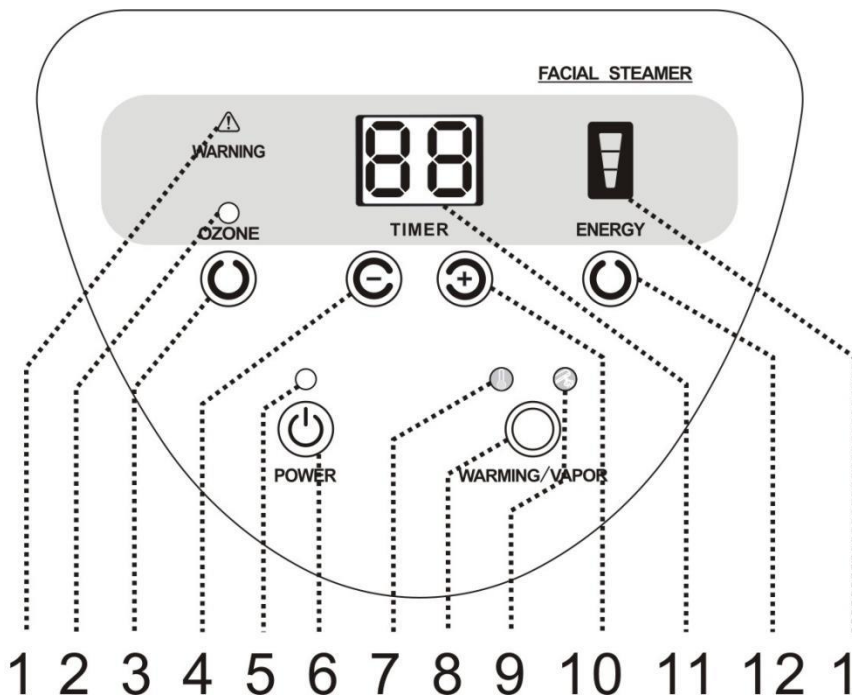

- 1.Low Water Warning Indicator
2. Ozone Working Indicator
3. Ozone Power Switch
4. Time Decreasing Key
- 5.Power Indicator
- 6.Power Switch
7. Warming Working Indicator
8. Warming or Hot Vapor turn on Key
9. Hot Vapor Working Indicator
10. Time Increasing Key
11. Time Display Window
12. Energy Selection Key
13. Energy Display Window

control panel operation:

1.Plug the power plug.

2.“”power indicator on of the power key “” when turn on the power key “”, motor goes on standby status.

—Time window displays “15”minutes.

— “”on the energy window shown the largest gear.

—Ozone working status indicator“”on the “”ozone switch is off.

—Spray working status indicator “”or“”on the “”WARMING/VAPOR switch is off.

3.Turn on the hot spray key” **WARMING/VAPOR** ” switch

Hot Vapor function: Turn on the “**WARMING/VAPOR** ” switch, spray working status indicator “”on the “”lights. At this time, the host begins heating, steam spouts from the nozzle around six minutes.

Warming function: Turn on the “**WARMING/VAPOR** ” switch, **WARMING** working status indicator “”on the “”lights. At this time, the host begins heating.

4.Time setting

Press key “” and “” for TIMER" to set up the working time according needed.

When spray working status indicator“”, time range 1~30 min, default 15 min.

When Warming working status indicator“”, time range 1~90 min, default 30 min.

5. Energy setting

Press “**ENERGY**”  to select the output of the mist, there are high, middle and low mist for selection; the “” display window lights full means the highest energy, contrary means the lowest energy. Before the mist spouts should be selected the highest level in using, then adjust the mist according needed.



6. Turn on the ozone function

Press the **ozone** switch “”, the ozone working status indicator “” is on; At this time, ozone pipe turns on, obviously ozone smells in the steam with distinct atomization phenomenon.

7. Water level detection function

The host machine detect the water level automatically when heating “” or “”, when the water level under the normal working (water level between the maximum and minimum), the host machine will continuous heating ; When too high or too low water level is detected , (when the water level was too high, the time display window will display "E2", when the water level too low, the time display window will display "E1", the host machine turn off the heating right away.

At this time, the indicator “” or “” on “**WARMING/VAPOR**”  key is off. the buzzer will give the alarm signal "De De" Continuously until the water level reach the normal water lever., and unlock the all the locked button, at this time, press the " **WARMING/VAPOR**”  to rework

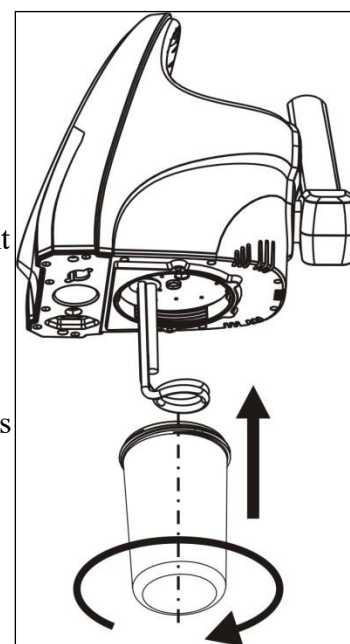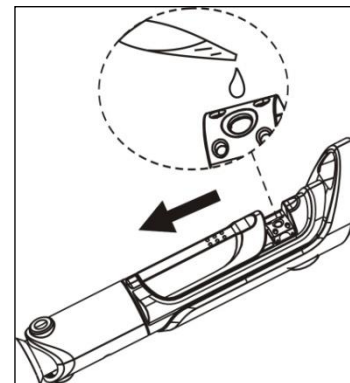

Spray arm and spray head setting:




Spray arm with special design. It can adjust by different angle to fit different environment. As the follow picture:

1. 130 degree can be adjust by left-right direction.

2. 60 degree can be adjust by forward-backward direction.

3. Spray head can be rotated 360 degree.

Remark: Do not adjust with overexertion, it is easy to damage the joint inside.

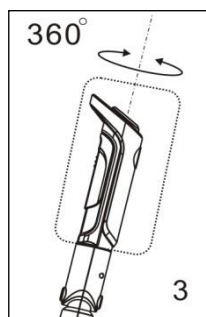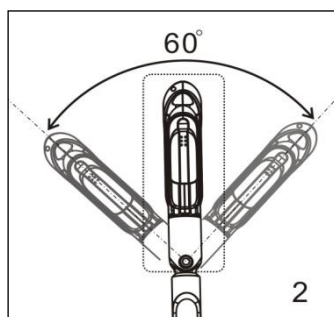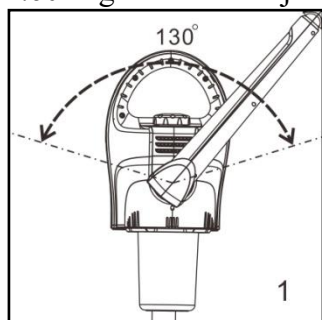

Adding essence oil:

If there's need essential oil spray therapy, open the essential oil sliding cover on the top of the nozzle, and use burette drop essential oil into the cotton pallet (one or two drops of essential oil are enough). (As right picture shown)

If it needs to change the essential oil, take away the essential oil base with tweezer and change the cotton. Reset it and add new essence oil.

Water cup installation and removal:


Cut off the power before removing the water cup. Please water cup cold and then take it apart. Holding the host machine in one hand, and rotate clockwise by holding the bottom of the cup in another hand.

Vice versa, cut off the power before install the water cup. Holding the host machine in one

hand, and rotate counterclockwise by holding the bottom of the cup in another hand. (As picture shown)

Panel instructions:

1. Pause indicator
2. Start/Pause
3. High-Frequency electrotherapy function indicator
4. Spray, vacuum function indicator
5. Ultrasonic wave function indicator
6. Function selection key
7. Time display window
8. Time increase key
9. Time decrease key
10. Extremely sensitive skin indicator
11. Sensitive skin indicator
12. Normal skin indicator
13. Skin selection key
14. Continuous wave working indicator
15. Hybrid wave working indicator
16. Broad pulse wave working indicator
17. Waveform selected function indicator
18. Power indicator
19. Power Switch
20. Galvanic zwitterions conduction function indicator
21. Brush function indicator
22. Skin scrubber function indicator
23. Energy display window
24. Energy decrease key
25. Energy increase key
26. Cation conduction working indicator
27. Anion conduction working indicator
28. Ion conduction selection key
29. Eyes probe working indicator
30. Face probe working indicator
31. Probe selection key

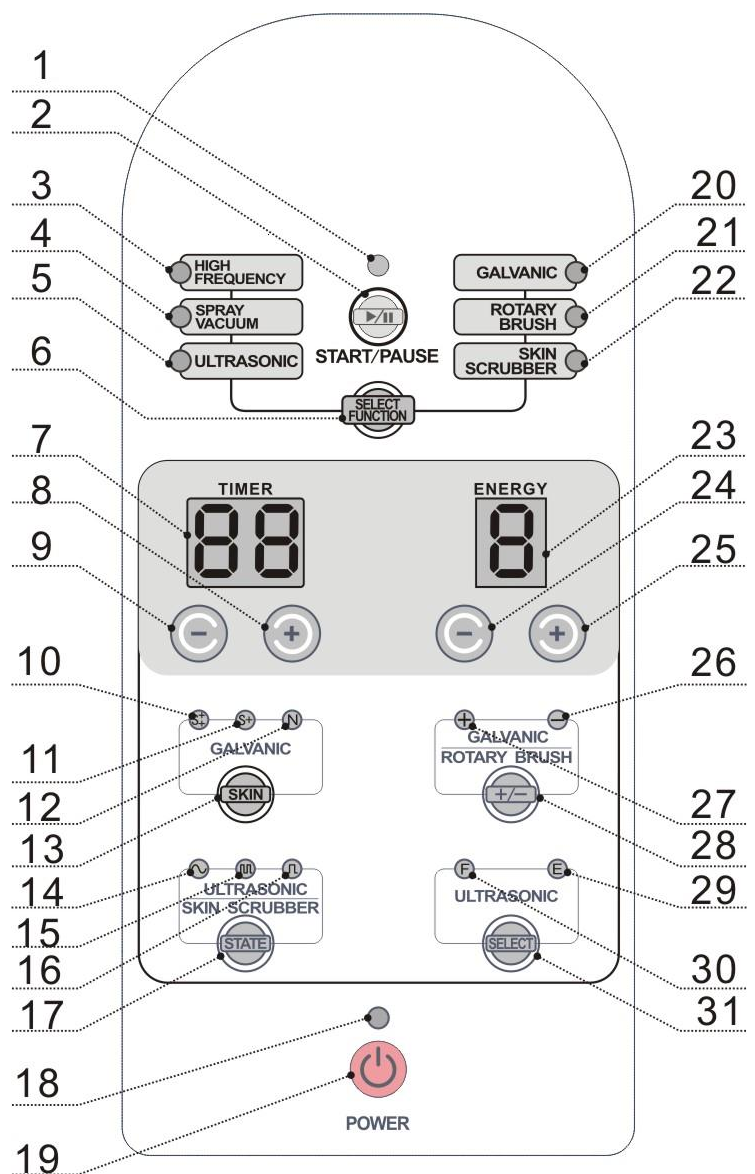

●High-Frequency Electrotherapy Function Adjustment:

Under the status of stand-by, press "SELECT FUNCTION" to


select High-frequency electrotherapy function. The indicator "HIGH FREQUENCY" lights, the host machine enters the adjustment status; under this status, it is ok to adjust as demand.



After setting, press “” key to turn on host machine output and next to beauty operation.

Direct Electrotherapy :

1. The beautician holds the handle of high-frequency electrotherapy (see Fig. right), **DO NOT** contact the glass electrotherapy tube with hand or metal matter, switches on the power of host machine, and adjusts "ENERGY" to min. Level.
2. Contacts the cured part of client with glass electrotherapy tube (see Fig. right side), and presses "" till the indicator  lights, slowly press "ENERGY"  key to adjust the energy output and the glass tube will generate ionization lighting until there is output of warn high-frequency current. When face creams or face mask is being used by the client, moves slowly the glass electrotherapy tube from neck toward chin, face, cheek, nose and forehead. During the course, adjusts slowly the "ENERGY" level as demand. When the heat is generated face skin, it has effectiveness of appeasement and sedation by directly using high-frequency electrotherapy. Meanwhile it has bactericidal effect, and is suitable for oily and acne skin.
3. After the operation, firstly slowly adjusts down the "ENERGY", presses "" key till pause indicator "" lights, the output of host machine pauses. And then takes away the glass electrotherapy tube contacting with face.

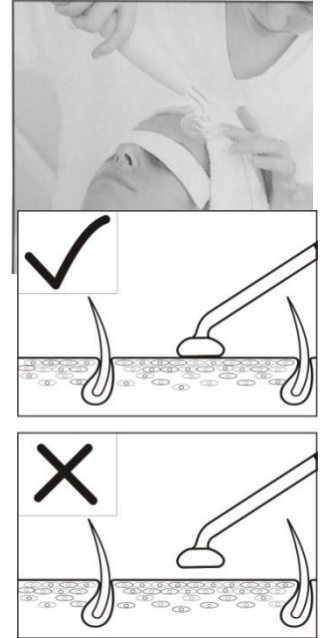

The Functions of Direct Electrotherapy:

1. Warm vibration is good to appease the skin.
2. Stimulates the circulation of blood, provides nutrition for skin to improve gloss and elasticity of skin.
3. Promotes cell metabolism, helps to breath and excretion of skin.
4. Heating effect on the skin will increase the cell permeability, and helps to improve the absorption ability of skin and enhance the treatment effect.

Indirect Electrotherapy:

Firstly the client holds the straight-bar electrotherapy tube, switches

on the power, then presses "", adjusts "ENERGY" to level as demand, the beautician give a massage to treatment part. During the whole course, the beautician does not contact with electrode, and before take away the electrode bar, must press "" till pause indicator "" lights, and operates after the output stops.



Functions of Indirect Electrotherapy:

1. Stimulates the circulation of blood, promotes cell metabolism.
2. Adjusts and recover the absorption of nutrition and resistance to bacteria.
3. Enhances the activity of skin basement cell by current effect, to improve the massage effect.



Notice for High-Frequency Electrotherapy Operation:

1. Squeak sounds during the normal working of electrotherapy handle.
2. When direct electrotherapy is used, the spark may spray when the glass electrotherapy tube keeps a distance less than 1 inch with skin, and it would simulate the skin which could make uncomfortable. Therefore, the energy level should be adjusted to the minimum one before contacting with skin. If the spark is felt, a dry cotton yarn can be used to cover the face

(Refer to the Fig.).

3. The electrode tube should be tightly closed to the face without any gap if direct electrotherapy is available, otherwise, it is easily to generate spark and to stimulate the skin.

4. For treatment of face, 8 min. for oily skin while 3~5 min. for dry skin. The current intensity is stronger for oily skin than for dry skin.

5. When operation with indirect electrotherapy, the beautician should keep one hand on face of the client, to prevent cut-off of current impacting the effect.

6. To prevent electric shock, firstly presses "▶/||" before the glass electrotherapy tube contacts with skin, then slowly increases the energy.

7. When takes away the glass electrotherapy tube, firstly adjust the energy to the minimum level, and presses "▶/||" to shut down the output, till the pause indicator "●" lights, then takes away.


8. "POWER" must be turned off when installs and uninstalls the glass electrotherapy tube.

9. Each glass electrotherapy tube must be kept clean, dry and sterilized before and after used.

10. The electrotherapy handle can be cleaned by towel, DO NOT clean in the water, if it is contaminated by oily dirt, it can be cleaned with absolute alcohol, and kept in dry.

Spray and vacuum function:

Spray function:


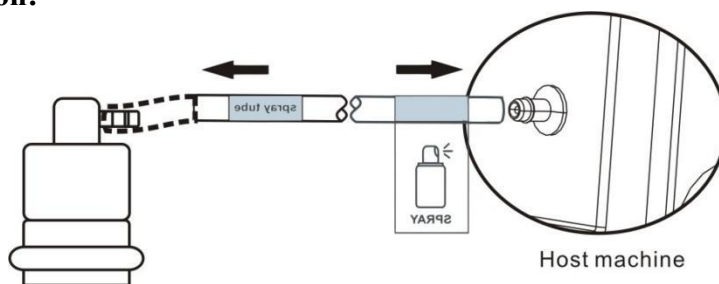


Firstly checks if the flexible tube of spray bottle connects appropriately with interface between host and spray bottle (see the Fig. right); switches on the power of host, adjust working time and energy level, and the host is under condition of pause output.


Under start up of host output, hold tightly the spray bottle by hands, and presses slightly the vent at top of spray bottle by fingers, then the mist will spray from the mouth; two spray bottles are prepared, for different liquids, and the bottle could be switched alternatively based on demand; the host should be pause for output when switch the spray bottle.

It should be careful for treatment of critical parts e.g. Mouth, nose, eyes. Please close the eye, mouth to prevent liquid sucked into the naris and should spray out toward below of chin, and keeps certain distance with face. After spray, pauses the host, and absorbs remained liquid on surface with cotton pellet or soft paper.

Vacuum function:

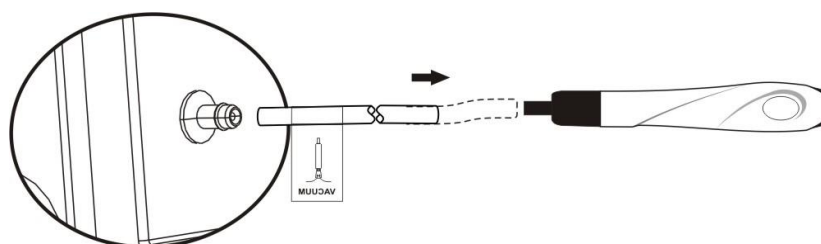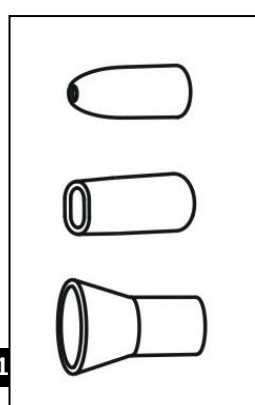

Firstly checks if both ends of vacuum tube connect

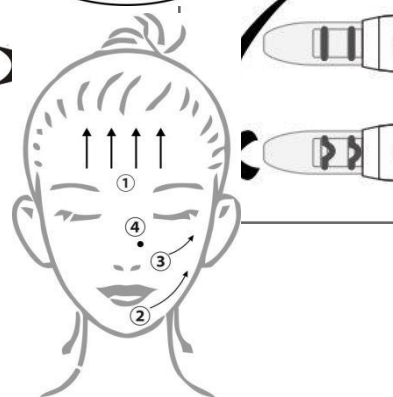

appropriately with corresponding interface (see the Fig. right); turn-on the end of vacuum bar to check if the sponge filter is available; if no, fill in the sponge filter and fix the vacuum bar; connects the front-end of vacuum bar with transparent sucker. Please be attention when installation, you should rotate the head slowly while insert to make sure the gasket ring keep standard shape, air tightness as picture shown. Switches on the power of host, adjust working time and energy level, and the host is under condition of pause output.


Under the start up of host, hold tightly the vacuum bar by hands (like holding a pen), moves the bar to working location, the nozzle of transparent sucker should contact with skin, and presses slightly the relief-pressure air vent, then the skin contacting with sucker is absorbed, and moves around the bar on related parts, where allows to absorb black head, acne in skin. Different suckers should be selected based on the different skin areas:

1. Round let mouth sucker is used for blackheads, acne in small area skin;
2. Flat mouth sucker is used for blackheads, acne in narrow area skin;
3. Horn mouth sucker is used for blackheads, acne in big area skin.

●Daily Maintenance

1. The flexible cable of probe and host machine must be tidy up. **DO NOT** twist.
2. Be careful to use the probe. The probe should be kept clean and dry; And sterilize the probe with ethanol or other agents after usage.
3. Each glass electrotherapy tube must be careful use and kept clean, dry and sterilized before and after used.
4. The tightness of electrotherapy tube should be check frequently, test method: puts the tube to be tested into the handle, and places the handle stably on the table, connects with cable of host, starts up the host, and increases gradually the energy output, it is normal to observe the lighting in electrotherapy tube.
5. The sucker should be kept clean, cleans and sterilizes after using and appropriately stores.
6. The nozzle of spray bottle should be kept in clean and unblocked.
7. To protect the flexible tube from damage so as to keep the working effect.
8. Periodically wipes the instrument with dry dishcloth and to place in dry, ventilation area, the cable of instrument should be place in order but not twist.
9. The instrument should be wiped with dry dishcloth, **DO NOT** wet in water.
10. Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
11. The instrument should **NOT** be placed in sunlight, raining, moisture, dust and smoke accumulated area.

Ultrasonic function:

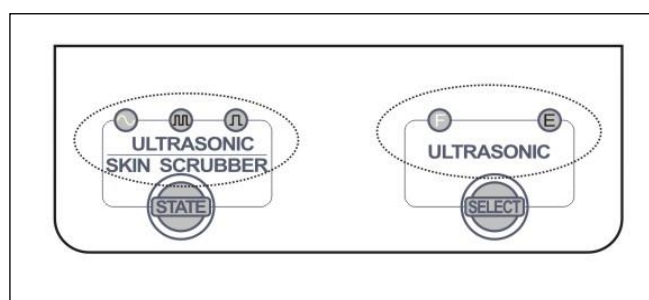

Under the status of stand-by, press "  " to select ultrasonic wave function. When the ultrasonic wave function is selected, the indicator "  " lights, the host machine enters the adjustment status for ultrasonic wave; time display "15" min., energy display " 5", probe selection key "  ", on the left "


"F" face probe indicators, waveform selection key "STATE" on the left "continuous wave indicators. Under this status, it is ok to adjust time and energy as demand, also the probe and waveform.

Output probe selection: press "SELECT" key for ultrasonic wave control to select the output probe. The indicator "F" lights meaning to the face probe is selected currently; indicator "E" lights meaning to the eye probe is selected currently.

Waveform output selection: press "STATE" to select output of waveform. The indicator "continuous wave lights meaning to continuous wave output; if the indicator "hybrid wave lights meaning to hybrid wave output; if the indicator "board pulse wave lights meaning to board pulse wave output.

Under adjustment completes, press " to start or pause the output from host machine.

Galvanic Function:

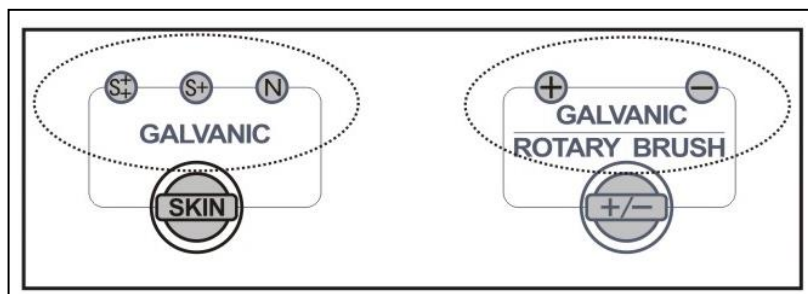

Under the status of stand-by, press "SELECT FUNCTION" key to select Galvanic function. The indicator "GALVANIC" lights, the host machine enters the adjustment status; under this status, it is ok to adjust as demand.

Skin type selection: presses "SKIN" to select appropriate type of skin. The indicator "N" lights meaning to normal skin type is selected; indicator "S+" lights meaning to sensitive skin type is selected; indicator "S+" lights meaning to extremely sensitive skin type is selected.

Ion conduction polar selection: presses "+/-" to select output polar for probes with red flexible cable. The indicator "+" lights meaning to cation conduction working state; indicator "-" lights meaning to anion conduction working state.

Start or pause output: under the status of stand-by or adjustment completes, press " to start or pause the output from host machine; when the host is starting output, the start up working indicator " lights; when the host is pause or stand-by, the pause indicator " lights.

There are three operation methods based on different treatment effects of zwitterions conduction function:

1、single-polar conduction method by the beautician

(1) Coat directly with half of plant essence on skin, and the other soak the cotton pellet and rolled on the probe.



(2) Switch on the power of host and presses "POWER", adjusts "ENERGY" to min. Level, let the client

hold the electrode bar while the beautician places the probe on neck of the client, presses "▶/||",



starts up the host, then presses "⊕/⊖" to adjust for "" cation working state and adjusts energy output



appropriately via "ENERGY" key  or  (A slight stinging sensation is felt).

(3) The probe moves slowly by a shape of "Z" on face or part of neck with keeping touch of skin during the whole course of duration at 3~5 min.



(4) After operation, presses "▶/||" to pause the output, switch on "POWER", then takes away the probe as well as the electrode bar hold by the client.




2、Impurity Export

When the operation of impurity export, the moving probe should be adjusted to negative electrode while the electrode bar or probe hold by the client should be connected with positive electrode. Uses special essence for impurity export, or saline to soak the cotton pellet and rolls the probe; or cover a thin layer cotton yarn or face mask soaked with essence on treatment parts. Then presses "⊕/⊖" to adjust to "⊖" anion conduction working state, and other steps are the same as for nutrition conduction.

●Rotary brush cleaning

function

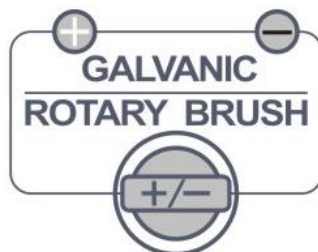


Under the status of stand-by, press "SELECT FUNCTION" to select Rotary brush cleaning

function. The indicator "ROTARY BRUSH" lights, the host machine enters the adjustment status; under this status, it is ok to adjust as demand.

Time display "15";

Energy display "5";

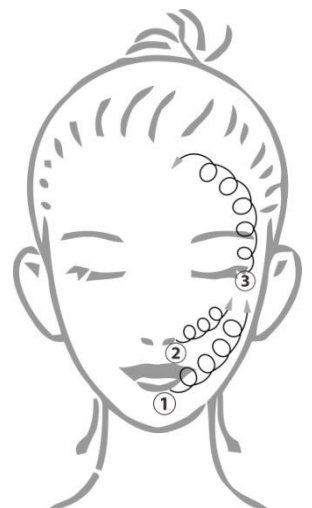

"⊕/⊖" clockwise swirling output status indicator at left side "⊕" on;

"▶/||" start indicator at right side "●" off.


Under this status, the volume of select function working time and energy output can be adjustable, and also can be selected the swirling direction of brush.

Brush swirling direction select: Press  key to select output status of brush. Indicator  is

clockwise swirling output status; and indicator  is inverse hour swirling output status. Changing swirling direction on the brush working status, the host machine will come to protection status. It will switch automatically until the brush stop to Swire. After setting well, press  to turn on host machine output, then can be come to beauty operation.


Change brush:

1. Unload brush: Hold the brush handle in one hand, and hold the plastic part of the brush by another hand, then pull in opposite direction, and then unload the brush easily by hearing “Ka” sound.
 2. Setting up brush: Hold the brush handle in one hand, and hold the plastic part of the brush by another hand, then rotate and push in opposite direction, and then set up the brush easily by hearing “Ka” sound.
- For unload or set up brush, only can be push or pull the plastic part by hand, but not to do this to the material of brush (e.g. EVA or fuzz etc.). Otherwise will make bad effect to work, even destroy brush.

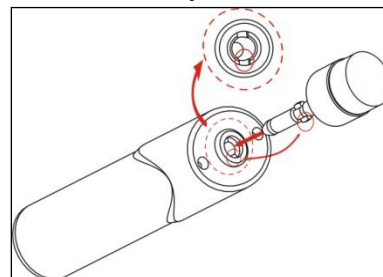

Cleaning skin:

1. Firstly use sprayer or clean water softens skin.
2. Wipe skin care production (e.g. cleaning milk or cleaning jelly etc.)equably on skin, according to the characteristic and need to cleaning skin area to choose suitable brush and set up on the brush handle, dip a little water on brush and move it to needed to cleaning skin, turn on host machine then can be come to cleaning and massage.
3. Keep skin moist during the whole cleaning operation.
4. After cleaning with brush, cleaning the skin care production with towel or facial tissue, and then cleaning skin with moist towel and facial cotton.

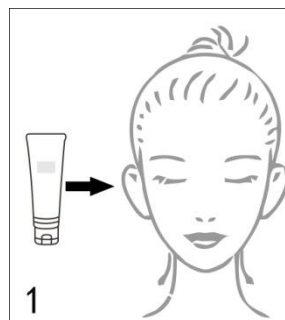

Using attention and notices of Brush function:

1. Before using this brush to work, please check if brush was insert in the brush handle firstly.
2. Please turn down the rotary speed of brush before touching customers' skin; when touch customers' skin, it must be slightly to avoid causing customer feel uncomfortable.
3. When use brush to cleaning or massaging skin, please use thickness skin care products and turn down rotary speed of brush, in case of skin care products splashed by fast rotary speed and causing waste and pollution.
4. If it is needed to change brush during skin caring, please stop outputting of the host machine. And wait brush stop to unload and setting up.
5. Please clean and sterilize all the used brushes after finishing work, and keep them in the dry place.
6. It cannot clean brush handle by water but wipe it clean by towel and keep it in the dry place.

Skin Scrubber function:



Under the status of stand-by, press "SELECT FUNCTION" to select Skin Scrubber function. The indicator "



" lights, the host machine enters the adjustment status; under this status, it is ok to adjust as demand.

Control status: Time display "15";

Energy display "5";

Continuous wave indicator " " on;



" " start indicator above " " off.

Under this status, press " " to start or pause the output for host machine.

3、Function setting: to adjust as demand under stand-by status.

Time setting: press " " and " " to set up the working time of host machine, time range 1~30 min.

Energy adjustment: press " " and " " to control the energy output of host machine, adjustment levels 1~9.

Waveform output selection: press " " to select output of waveform. The indicator " " lights meaning to continuous





wave output; if the indicator " " lights meaning to hybrid

wave output; if the indicator " " lights meaning to board pulse wave output.

After finished setting , press " " to start facial treatment.

2、Cleaning skin:

- (1)、Use sprayer or cleaning water softens skin to be cleaned.
- (2)、Wearing the sensor on the wrist of customers.
- (3)、Press ENERGY key to adjust energy to suitable strength.
- (4)、To wipe skin care products on the skin surface evenly (like cleaning milk etc.), in the whole process of cleaning skin care to be kept moist.
- (5)、Reference to the presentation of A.B.C.D

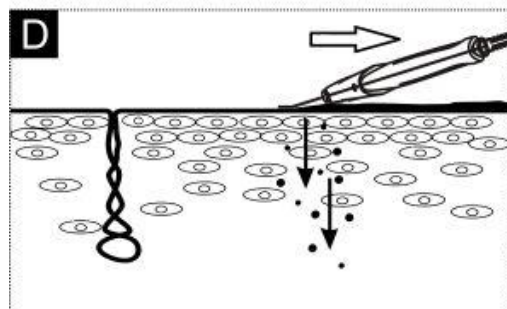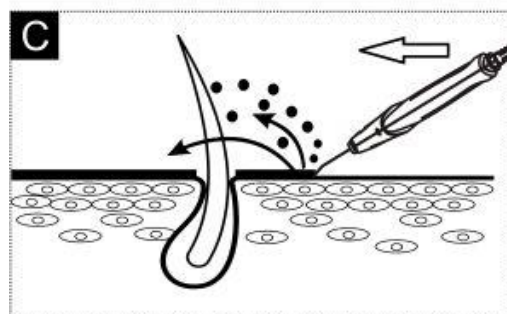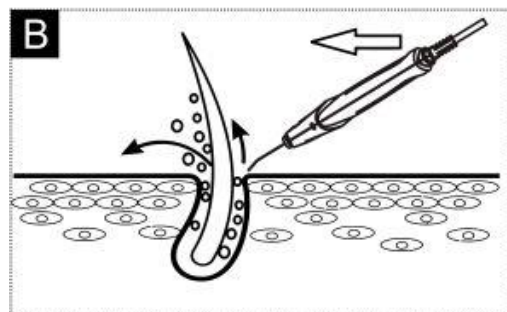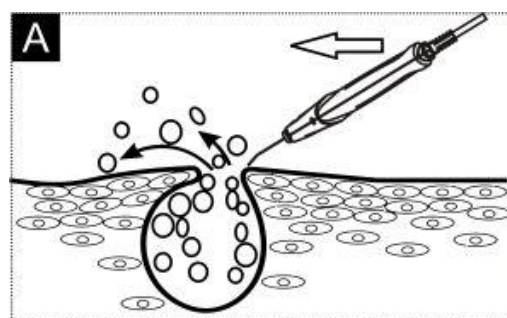

A、Eliminate acne without pain(As picture A).

Ultrasonic can be split the mature acne into smaller particles in a short time by vibration, and the whole process is painless.

B、Clean-up eruption(As picture B)

With 25,000 times vibration frequency can be cleared the eruption in the pores and residue of cosmetic very quickly.


C、Cleaning skin(As picture C)

Scraper knife and be cleaned the dirty things and aging skin.

D、Import massage(As picture D)

Place the head of the scraper knife on the surface skin flatly, and with push-pull can be imported essence to skin effectively(As picture D).

●Operating instructions:

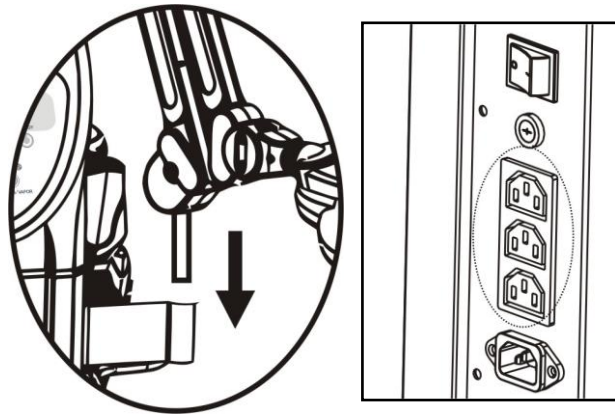

1. Put the magnifying lamp in the host machine. (As picture shown at right side)
2. Connect the plug to the socket correspondingly. (As picture shown at right side);
3. Open anti-dust cover and adjust to the correct position by adjustment joint.
4. Turn on the power switch and start beauty operation.

●Specifications and Technical Data:

Product Name	9 IN 1 beauty instrument
Item No.	H5050
Function	facial steamer、 magnifying lamp、 ultrasonic、 skin scrubber、 galvanic、 high-frequency、 rotary brush、 spray and vacuum
Rated Voltage/Power	AC 100-120V~ AC 220-240V~
Working Frequency	50/60Hz
Rated Power	800W/850 W
Dimension (L*W*H)	1200*440*550mm
N.W	22.5 Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.



Imported by
WEEKKO BARCELONA SL
Pol. Ind. La Valldan

Cami de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com