



PLANT

Electric Podiatry Chair
Ref. WKP005.3

manual

Upmarket podiatry chair equipped with 3 motors that control height, backrest and seat inclination. Reclining armrests and headrest cushion adjustable in multiple fixed positions. Legrests swivel in two fixed positions up to 90° and are independently adjustable in height by gas lift. White or grey PU upholstery of high quality that eases cleaning.



**podiatry
& pedicure**

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the bed. If an inspection becomes necessary, please contact your supplier or an authorised technician.
- Never use the bed in wet or poorly ventilated environments, such as bathrooms to prevent from damaging the electrical components of your bed.
- Never use the bed outdoors; the exposure of the elements could damage the bed or reduce its lifespan.
- If the power cord was damaged, please ask the dealer to replace it.

CE-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

RECYCLING



Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose it in accordance with local laws for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.


WELCOME!

Thank you very much for your purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation. The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

TECHNICAL PARAMETERS

weights and sizes	packing size (cm)	137x63x56
	product size (cm)	168x81,5x54/128
	highest position (cm)	128
	lowest position (cm)	54
	backrest angle	0° - 80° ±10%
	legrest angle	0° - 80° ±10%
	swivel legrest angle	0° - 90° ±10%
	seat inclination	0° - 12° ±10%
	headrest extension (cm)	18
	legrest extension (cm)	10
	net weight (kg)	75
	gross weight (kg)	86
electrical characteristics	rated voltage	100 ~240V
	frequency	50Hz/60Hz
	power	200W
	output	29VDC, Max 4.5A
other characteristics	environment temp.	10 ~ 40°C
	relative humidity	30 ~ 85%
	max. loading capacity (kg)	180


TECHNICAL SHEET

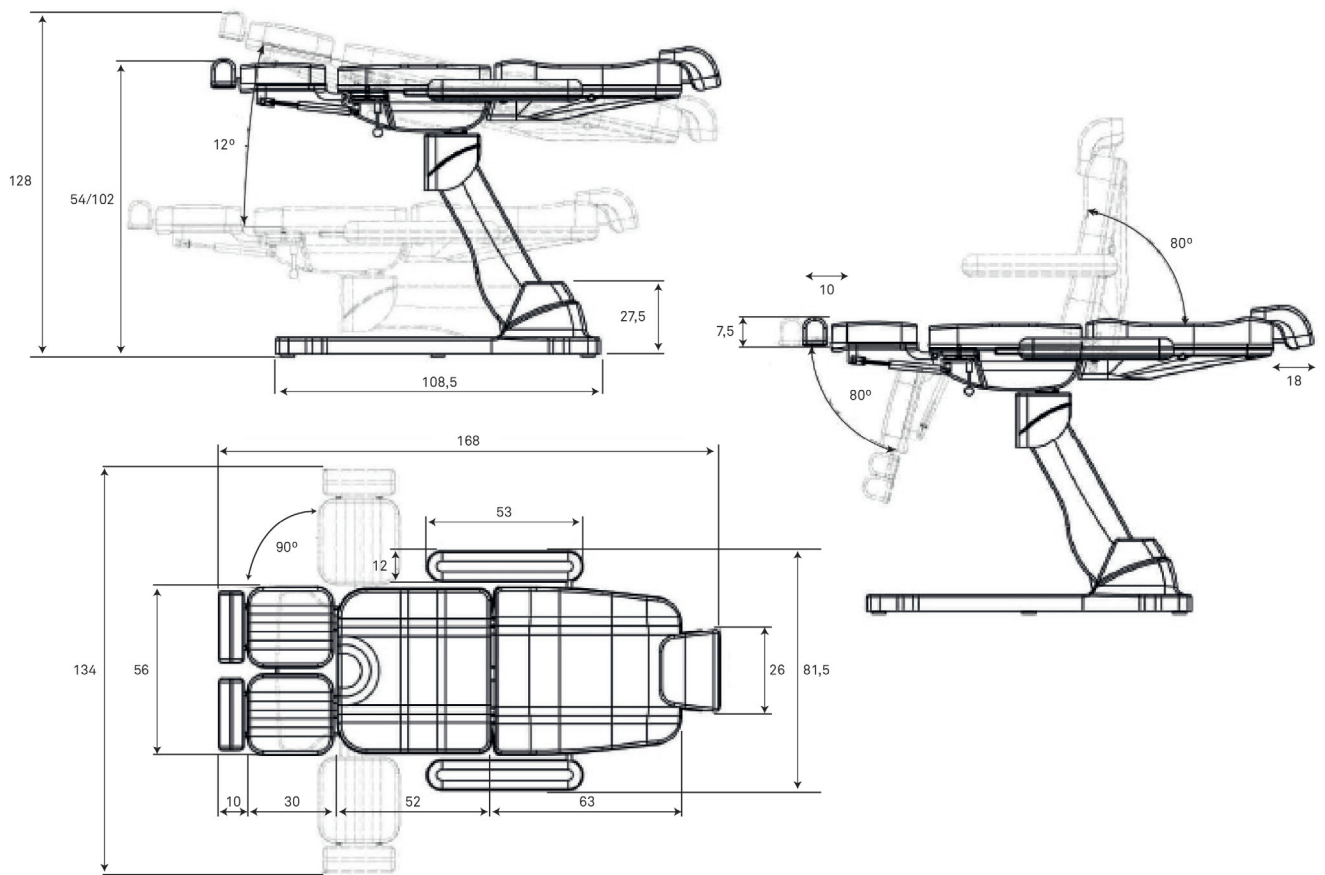

ASSEMBLY AND INSTALLATION

1. Unpack the carton, take out all accessories and check all the parts are correctly included.
2. Connect power and press the button  of remote control to rise the backrest, then pick up the right armrest to aim at the hole in the right side of the backrest, to insert the armrest completely and turn it 180° to finish the right armrest installation. Repeat this action to install the left armrest.
3. Insert the headrest into the top of the backrest.
4. Connect power and press the button  of remote control to decline the backrest and press the button  to rise the foot part in order to make the whole bed flat. Then put the bed in a suitable place and ensure there is around 50 cm behind the bed and in front of the bed.


OPERATION

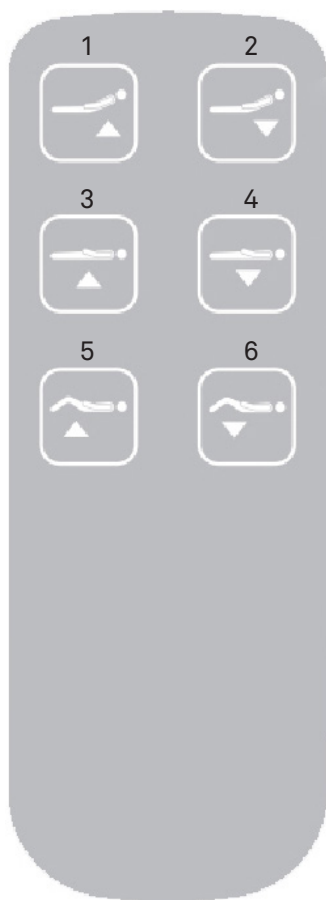

1. Standard configuration: Hand controller

	Description	Operation
1	backrest angle button	Backrest goes up by pressing the button.
2	backrest angle button	Backrest goes down by pressing the button.
3	bed height button	The bed height rises up by pressing the button.
4	bed height button	The bed height declines by pressing the button.
5	seat's tilting button	The seat tilts back by pressing the button.
6	seat's tilting button	The seat goes back to the zero position by pressing the button.

1. Optional configuration: Foot controller

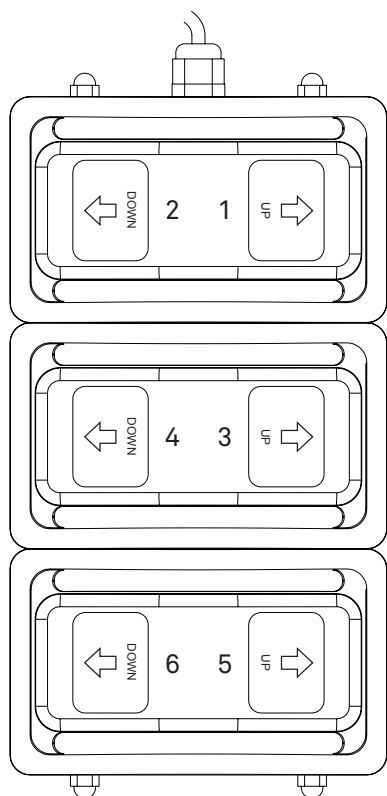

	Description	Operation
1	backrest angle button	Backrest goes up by pressing the button.
2	backrest angle button	Backrest goes down by pressing the button.
3	bed height button	The bed height rises up by pressing the button.
4	bed height button	The bed height declines by pressing the button.
5	seat's tilting button	The seat tilts back by pressing the button.
6	seat's tilting button	The seat goes back to the zero position by pressing the button.

TROUBLESHOOTING

Please follow the next counter measures in case of problem. If the problem still persist, stop using the product immediately to avoid any unforeseen damage or injury. Disconnect the power plug from the main outlet socket and consult the authorised sales agent. Do not attempt to repair or modify the equipment at home.

Problem	Solution
The remote control is not working even though the power supply is connected.	1. Check whether the plug is properly inserted into the outlet and in good condition. 2. Insert other appliance into the socket to check whether the socket has power.
There is noise during the bed operation.	1. Please onfirm where the noise comes from. If it comes from the mechanical structure, please add lubricating oil. 2. If the noise persists after having lubricated the axis and gaslifts, please contact the supplier.
Control box has reaction, but motor does not work.	Check whether the remote control is faulty.
Headrest, footrest, armrests are difficult to pull or push.	Please clean the steel pin and add some clean wax.

PRECAUTIONS

- Be careful not to damage the table when you take it out from the box.
- Once the chair is placed in the work space, make sure that the power cord is well situated to not hinder the chair performances and the professional's work.
- The frame is designed to take full advantage of the table. The table must not be used for other purposes than podiatry and pedicure treatments.

CLEANING

- Before cleaning the bed, unplug the power cord.
- The frame should be cleaned with a dry cloth or with a slightly damp cloth for the toughest stains.
- Clean the upholstery with a clean and white cloth and water with a gentle soap.
- Avoid abrasive detergents, use only neutral products.

weelko.com

Barcelona / Showroom

Pol. Ind. La Valldan
Cami de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 90 20
E. info@weelko.com